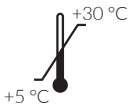
UN 1903

DENTAL USE

GREEN&CLEAN M2

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EN – Instructions for use

Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images: The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. General instructions

1.1. Storage temperature

+5 - 30 °C

1.2. Product identifiers

GREEN&CLEAN M2 green: Subtilisin [9014-01-1], Edetic acid [60-00-4], Benzalkonium chloride [61789-71-7], Dioctyldimethylammonium chloride [5538-94-3], N-(3-Aminopropyl)-N-dodecylpropane-1,3,-diamine [2372-82-9].

GREEN&CLEAN M2 red: Subtilisin [9014-01-1], Edetic acid [60-00-4], Dioctyldimethylammonium chloride [5538-94-3], Dodecylguanidine mono hydrochloride [13590-97-1], N-(3-Aminopropyl)-N-dodecylpropane-1,3,-diamine [2372-82-9]

1.3. Explanation of the symbols

1 - CE mark with notified body; **2** - Medical device, **3** - Unique identifier of a medical device; **4** - UDI marking with standard compliant HIBC data content; **5** - Item number; **6** - Batch designation; **7** - Expiry date; **8** - Manufacturer; **9** - Name and address of the authorized representative's registered office in Switzerland; **10** - Observe instructions for use; **11** - Temperature limit; **12** - Quantity information; **13** - Health hazard; **14** - Corrosive; **15** - Dangerous for the environment; **16** - Harmful to health; **17** - Use hand protection; **18** - Use eye protection; **19** - Protect from heat / Protect from sunlight; **20** - Protect from moisture / Store in a dry place; **21** - Fragile; **22** - Package orientation at the top; **23** - Limited quantity.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of

METASYS Medizintechnik GmbH.

2. Intended use

GREEN&CLEAN M2 is a dual, actively defoaming concentrate for cleaning, disinfection and removal of unwanted odours in the suction lines of dental suction systems. **Indication:** Removal of blood and saliva as well as disinfection in the wastewater path of the dental suction system. To be used twice a day (at noon and at the end of the working day) and after surgical procedures. **Contraindication:** There are no contraindications related to specific patient populations or user groups. **Target group:** Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

DANGER: The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP). Harmful if swallowed. Causes severe skin burns and eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Very toxic to aquatic life. **GREEN&CLEAN M2 green:** Keep out of reach of children. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water (or shower). IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. In the event of respiratory symptoms: Call POISON CENTRE / doctor. **GREEN&CLEAN M2 red:** If medical advice is needed, have product container or label at hand. Keep out of reach of children. Do not breathe dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. Wear respiratory protection in the event of insufficient ventilation. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately

all contaminated clothing. Rinse skin with water (or shower). IF INHALED: In the event of breathing difficulties, remove the affected person to fresh air and immobilise in a position that facilitates breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor/... In the event of respiratory symptoms: Call POISON CENTRE / doctor. Dispose of contents/container to a suitable recycling or disposal facility.

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.2.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

General instructions: Immediately remove contaminated, saturated clothing.

After inhalation: In the event of breathing difficulties, give oxygen. In the event of irregular breathing or respiratory arrest, initiate artificial respiration. Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label.

After skin contact: In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.

After eye contact: In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist.

After ingestion: Rinse mouth immediately and drink plenty of water. Do not

induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

3.5. Personal protective equipment (PPE)

Eye/Face protection:

Suitable eye protection: Tightly sealing safety goggles.

Hand protection:

Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm. Breakthrough time: >480 min. Additional hand protection measures: Wear cotton undergloves if possible.

Product description

GREEN&CLEAN M2 is an actively defoaming concentrate for cleaning, disinfection and removal of unwanted odours in the suction lines of dental suction systems.

3.6. Composition

GREEN&CLEAN M2 green: Disinfection concentrate made of quaternary ammonium compounds and alkylamine.

GREEN&CLEAN M2 red: Disinfection concentrate made of quaternary ammonium compounds, alkylamine and guanidinium compounds.

4. Use

4.1. Dosage

The GREEN&CLEAN M2 product is used as a 1% solution (e.g. mix 6 ml GREEN&CLEAN M2 with 600 ml water).

Spectrum of action and exposure time	Standard	Test information	Exposure time	User concentration
bactericidal	EN 13727	high load	10 min	1.0 %
levurocidal	EN 13624	high load	10 min	1.0 %
tuberculocidal	EN 14348	high load	60 min	1.0 %
germ carrier test bactericidal	EN 14561	high load	15 min	1.0 %
germ carrier test levurocidal	EN 14562	high load	15 min	1.0 %
limited virucidal	EN 14476	high load	60 min	1.0 %

4.2. Application

GREEN&CLEAN M2 red and GREEN&CLEAN M2 green are always used alternately, i.e. as soon as the green disinfectant concentrate is used up, the red one is refilled and vice versa.

Filling and use of the GREEN&CLEAN M2 dispenser:

Cleaning and disinfection of the suction system and cuspidor is recommended 2x per day. Additional, special cleaning should also be carried out after surgical procedures.

A - After the dispenser is removed from the housing, the empty container is turned clockwise to remove it from the pump head. **B** - Then the concentrate is filled into the container. **C** - The filled container is turned counter-clockwise as far as the stop on the pump head. The dispenser is then placed back into the housing.

Daily use:

Cleaning and disinfection of the suction system and cuspidor is recommended 2x per day. Additional, special cleaning should also be carried out after surgical procedures. **D - Dispensing:** Actuate the dispenser pump 2x (= 6 ml GREEN&CLEAN M2 concentrate). **E - Mixing:** Take off the mixing container and fill up to the mark with cold water (= 600 ml 1% solution). **F+G - Suction:** Hold the small suction hose onto opening no. 1 (Fig. F) until air is sucked in (= 200 ml). Hold the large suction hose onto opening no. 2 (Fig. G) until air is sucked in (= 200 ml). **H - Cleaning and disinfecting the rinsing basin:** Empty the remaining solution (= 200 ml) into the rinsing basin. Do not rinse (allow time to work, see table „Spectrum of action and exposure time“).

5. Disposal

Dispose of product in compliance with official regulations.

- before intended use:

Product waste code: 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

- after intended use:

Waste code Packaging 70699; Hazardous waste: No; Waste designation: Plastic packaging

6. Annex

6.1. Order numbers and scope of delivery

122000026 - GREEN&CLEAN M2 red/green 500 ml, 1 bottle each, dispenser

122000027 - GREEN&CLEAN M2 red/green 500 ml, 2 bottles each

122000028 - GREEN&CLEAN M2 red/green 500 ml, 1 bottle each

122000030 - GREEN&CLEAN M2 green 500 ml, 25 bottles

122000031 - GREEN&CLEAN M2 red 500 ml, 25 bottles

Application aids:

121000008 - AH GREEN&CLEAN, M2, dosing pump for dispenser

121000009 - AH GREEN&CLEAN, M2, dispenser

6.2. Change history

Revision ZK-55.111/00, 15.06.2022: Newly created.

Revision 200005323v01, 23.10.2024: Order numbers updated; pictograms GHS07 (exclamation mark), UDI and HIBC added; "Follow instructions for use" symbol replaced. Corrected various spelling and translation errors. Subject to printing and setting errors!

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Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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