# METASAN® P

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# EN – Instructions for use

## Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

**Images:** The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

### 1. Notes

#### 1.1. General instructions

Storage temperature: +5 - 30 °C

#### 1.2. Product identifiers

Hydrogen peroxide [7722-84-1]

Silver nitrate [7761-88-8]

#### 1.3. Explanation of the symbols

1	CE mark with notified body
2	Medical device
3	Unique identifier of a medical device
4	UDI marking with standard compliant HIBC data content
5	Item number
6	Batch designation
7	Expiry date
8	Manufacturer
9	Follow instructions for use
10	Temperature limit
11	Quantity information
12	Responsible for the translation
13	Use hand protection
14	Use eye protection
15	Protect from heat / Protect from sunlight
16	Protect from moisture / Store in a dry place
17	Fragile
18	Package orientation at the top
19	Stacking limit
20	Name and address of the authorized representative's registered office in Switzerland
21	Information

## 1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYs Medizin-technik.

## 2. Intended Purpose

Solution for disinfection and elimination of germs in water-bearing pipes in treatment units. **Indication:** Limitation of total bacterial count and elimination of Legionella pneumophila. **Contraindication:** Not applicable. **Target group:** Professionally trained healthcare professionals.

## 3. Safety-related information

### 3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

### 3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP). If medical advice is needed, have product container or label at hand. Dispose of contents/container to a suitable recycling or disposal facility.

### 3.3. First aid measures

**General instructions:** Change contaminated, saturated clothing. In case of accident or if you feel unwell, seek medical advice immediately (show safety data sheet if possible).

**After inhalation:** Provide fresh air.

**After skin contact:** Subsequently wash off with water.

**After eye contact:** Rinse immediately carefully and thoroughly with eye-bath or water.

**After ingestion:** Rinse mouth immediately and drink 1 glass of water.

### 3.4. Personal protective equipment (PPE)

**Protection and hygiene measures:**

Avoid contact with skin, eyes and clothing.

**Eye/Face protection:**

Framed glasses.

## Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericide against Legionella pneumophila	EN 13623	low yeast extract load	1 hour	10%
			24 hours	1%

### Hand protection:

Suitable gloves type: Single-use gloves.

Suitable material: NR (natural rubber (caoutchouc), natural latex).

Recommended glove brands: EN ISO 374

### Respiratory protection:

Respiratory protection is not required when used properly and under normal conditions.

### 3.5. Combination with other products / materials

METASAN® P is used in combination with treatment units from Dentsply Sirona®.

## 4. Product description

METASAN® P is a disinfectant concentrate for disinfecting water and decontaminating process water.

### 4.1. Composition

<1.5% hydrogen peroxide, less than 0.1% stabilizers and silver

## 5. Use

### 5.1. Dosage

The disinfectant METASAN® P is dosed automatically by the Dentsply Sirona treatment unit.

If a bottle solution is used, the following dosages should be used:

Final volume	Volume of METASAN® P	Volume of water
1 liter (1% solution ratio)	10 ml	990 ml
1 liter (10% solution ratio)	100 ml	900 ml

### 5.2. Application

Suitable for Dentsply Sirona treatment units. Please observe the instructions for use of the treatment unit.

### 5.3. Shelf life after opening

If stored at the temperatures specified in chapter 1.1, the product has a shelf life of 3 years.

## 6. Disposal

Disposal of product in compliance with official regulations.

before intended use

Waste code - 070600  
Product

Hazardous waste No

Waste designation Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products

after intended use

Waste code - 150102  
Packaging

Hazardous waste No

Waste designation Plastic packaging

## 7. Annex

### 7.1. Order numbers and scope of delivery

122000059 - METASAN® P, 6 x 1000 ml

### 7.2. Change history

Revision ZK-58.105/00, 08.06.2022: Newly created.

Revision 200005330v01, 27.07.2023: Updated: Intended use, order numbers.

Revision 200005330v02, 24.05.2024: Symbol „Follow instructions for use“, Target group: „Professionally trained healthcare professionals.

Revision 200005330v03, 30.01.2025: Information added: Chapter 5.3 Shelf life after opening.

Corrected various spelling and translation errors. 30.01.2025. Subject to printing and setting errors!

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## Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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**METASYS Medizintechnik GmbH**