
UN 1987

DENTAL USE

CE 0044

MD

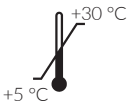# GREEN&CLEAN SD

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## EN – Instructions for use

### Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

#### Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

## 1. Notes

### 1.1. General instructions

Storage temperature: +5 - 30 °C  
UN 1987

### 1.2. Product identifiers

Ethanol [64-17-5], 2-propanol [67-63-0], Glyoxal [107-22-2], Didecylidimethylammonium chloride [1713-51-5]

### 1.3. Explanation of the symbols

**1** - CE mark with notified body; **2** - Medical device; **3** - Item number; **4** - Batch designation; **5** - Name and address of the authorized representative's registered office in Switzerland; **6** - Unique identifier of a medical device; **7** - UDI marking with standard compliant HIBC data content; **8** - Expiry date; **9** - Manufacturer; **10** - Temperature limit; **11** - Quantity information; **12** - Health hazard; **13** - Highly inflammable / highly flammable; **14** - Responsible for the translation; **15** - Observe

instructions for use; **16** - Use hand protection; **17** - Use eye protection; **18** - Protect from heat / Protect from sunlight; **19** - Protect from moisture / Store in a dry place; **20** - Package orientation at the top; **21** - Limited quantity; **22** - Stacking limit; **23** - Information.

### 1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

## 2. Intended purpose

Solution for disinfection of alcohol insensitive surfaces of non-invasive medical devices.

**Indication:** Disinfection of alcohol insensitive surfaces of non-invasive medical devices..

**Contraindication:** Not for usage on materials made of polymethyl methacrylate (acrylic glass), leather and synthetic leather covers.

**Target group:** Professionally trained health workers.

## 3. Safety-related information

### 3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

### 3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

**WARNING:** Flammable liquid and vapour. Causes serious eye irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children.

### 3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.2.

### 3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

#### General instructions:

In case of accident or if you feel unwell, seek medical advice immediately (show operating instructions or safety data sheet if possible).

#### After inhalation:

Ensure fresh air.

#### After skin contact:

Then rinse with: Water and soap.

#### After eye contact:

Immediately rinse carefully and thoroughly with eye wash or with water.

#### After ingestion:

If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

### 3.5. Personal protective equipment (PPE)

#### Eye/Face protection:

Suitable eye protection: Tightly sealing safety goggles

#### Skin protection:

Single-use gloves. Suitable material: NBR (nitrile rubber). Required properties: liquid-tight. Penetration time (maximum wearing time): <8 h. Thickness of the glove material: 0.1 mm. Wearing time with occasional contact (splashes): <30 min.

### 4. Product description

Alcoholic spray and wipe disinfection.

#### 4.1. Composition

45% ethanol, 15% 2-propanol, 0.2% quaternary ammonium compounds, 0.2% glyoxal

### 5. Use

#### 5.1. Application of spray disinfection

Spray the undiluted solution directly onto the surface to be disinfected and keep it moist for the

duration of the exposure time.

### 5.2. Application wipe disinfection

Put the undiluted solution on a wipe. Then completely wet the surface to be disinfected and keep it moist for the duration of the exposure time.

### 6. Disposal

Dispose of product in compliance with official regulations.

#### - before intended use:

Product waste code: 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

#### - after intended use:

Product waste code Packaging 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

### 7. Annex

#### 7.1. Order numbers and scope of delivery

122000047 / 60030001 -

5x GREEN&CLEAN SD 750 ml, 2x spray heads

122000045 / 60030002 -

6x GREEN&CLEAN SD 750 ml

122000048 / 60030003 -

1x GREEN&CLEAN SD 5000 ml

122000046 / 60300001 -

2x GREEN&CLEAN SD 750 ml, 2x spray heads

121000010 / 40200005 -

AH GREEN&CLEAN, SD, spray bottle, empty

#### 7.2. Change history

Revision 200005327, 29.06.2023: Newly created.

Revision 200005327v01, 13.08.2024: In chapter 1.1 General information, pictogram „Observe instructions for use“ replaced.

Corrected various spelling and translation errors. Status: 13.08.2024. Subject to printing and setting errors!

Spectrum of action and exposure time	Standard	Test information	Exposure time
bactericidal	EN 13727	high load	15 sec
levurocidal	EN 13624	high load	15 sec
tuberculocidal	EN 14348	high load	1 min
4-field test (bactericidal, levurocidal)	EN 16615	high load	1 min
limited virucidal	EN 14476   EN 16777	low load	1 min

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### Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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