GREEN&CLEAN CL N

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EN – Instructions for use

Translation of the original operation instruction:

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C

1.2. Product identifiers

Alkylethercarbonic acid (53563-70-5), Protease (9014-01-1), Lipase (9001-62-1)

1.3. Explanation of the symbols

1 - CE mark; **2** - Medical device; **3** - Unique identifier of a medical device; **4** - UDI marking with standard compliant HIBC data content; **5** - Item number; **6** - Batch designation; **7** - Expiry date; **8** - Manufacturer; **9** - Name and address of the authorized representative's registered office in Switzerland; **10** - Observe instructions for use; **11** - Temperature limit; **12** - Quantity information; **13** - Health hazard; **14** - Corrosive; **15** - Harmful to health; **16** - Use hand protection; **17** - Use eye protection; **18** - Wear protective clothing; **19** - Protect from heat / Protect from sunlight; **20** - Protect from moisture / Store in a dry place; **21** - Fragile; **22** - Package orientation at the top; **23** - Limited quantity.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended purpose

GREEN&CLEAN CL N is a concentrate for cleaning and deodorizing the pipes of dental suction systems.

Indication: For removing blood and saliva in waste water pipes of dental suction systems. For using at the end of working days and after long surgeries.

Contraindication: Not applicable. **Target group:** Professionally trained healthcare personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

WARNING: Causes serious eye damage. May cause

allergy or asthma symptoms or breathing difficulties if inhaled. Causes skin irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF INHALED: Remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a POISON CENTER/doctor. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container to a suitable recycling or disposal facility.

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

General instructions: Immediately remove contaminated, saturated clothing.

After inhalation: In the event of breathing difficulties, give oxygen. In the event of irregular breathing or respiratory arrest, initiate artificial respiration. Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label.

After skin contact: In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.

After eye contact: In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist.

After ingestion: Rinse mouth immediately and drink plenty of water. Do not induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

3.4. Personal protective equipment (PPE)

Eye/Face protection: Suitable eye protection: Tightly sealing safety goggles. Recommended eye protection brands: DIN EN 165.

Hand protection: Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm. Recommended glove brands: EN ISO 374. Additional hand protection measures: Wear cotton undergloves if possible.

Physical protection: Suitable physical protection: Laboratory coat.

Respiratory protection: Respiratory protection is not required when used properly and under normal conditions.

4. Product description

4.1. Product description

GREEN&CLEAN CL N is a concentrate based on alkylethercarbonic acid, protease and lipase for daily cleaning of suction systems.

4.2. Composition

Alkylethercarbonic acid, Protease, Lipase

5. Use

5.1. Dosage

The product GREEN&CLEAN CL N is used as a 1.6% solution (e.g. 10 ml GREEN&CLEAN CL N to 600 ml water).

5.2. Application

Note:

- > The user solution is sucked in by the suction flow of the suction unit and should remain in the suction system for 60 minutes if possible. Longer exposure times are also possible (e.g. over the weekend).
- > Rinse the suction system with cold water after each treatment!
- > Wear personal protective equipment during application!
- > Adhere to the dosage!
- > Mixed solution is intended for immediate use!

Application:

24 - Open the sealing cap of the dispensing container on the top of the bottle. Fill the container with 10 ml of the concentrate by gently pressing the bottle. **Only dispense when the sealing cap is open.**

25 - Empty the 10 ml GREEN&CLEAN CL N into the mixing container. Fill the container up to the mark with cold water (= 600 ml 1.6% solution).

26 - Hold the small suction hose on opening no. 1 until air is sucked in (200 ml).

27 - Hold the large suction hose on opening no. 2 until air is sucked in (200 ml).

28 - Empty the remaining solution into the rinsing basin (200 ml). Do not rinse.

29 - Wait for action time (60 min) and then rinse with cold water.

Allow GREEN&CLEAN CL N to work overnight and over the weekend. If necessary (e.g. in case of high soiling), we recommend an additional application at noon.

6. Disposal

Dispose of product in compliance with official regulations.

> before intended use:

Product waste code: 070699; Hazardous waste: no; Waste designation: Waste not otherwise specified

> after intended use:

Packaging waste code: 070699; Hazardous waste: no; Waste designation: Waste not otherwise specified

7. Annex

7.1. Order numbers and scope of delivery

122000004 - GREEN&CLEAN CL N 1000 ml, 2 bottles

121000005 - AH GREEN&CLEAN, CL N, mixing container

7.2. Change history

Version ZK-58.102/00, 18.06.2021: Newly created.
Version ZK-58.102/01, 28.06.2022: Correction: Chapter 1.2 „REF number“ changed to „item number“.

Version 200005319v01, 11.03.2025: Order numbers updated; UDI and HIBC pictograms added; „Observe instructions for use“ symbol replaced.

Version 200005319v02, 13.04.2026: „Dental use“ deleted. Target group updated (only in English).

Version 200005319v03, 29.04.2026: Version number corrected.

Corrected various spelling and translation errors. Subject to printing and setting errors!

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Traduction de la notice originale:

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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