GREEN&CLEAN ID N

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH
Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.


1. Notes

1.1. General instructions

UN 1903
DENTAL USE

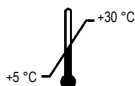Storage temperature

Product identifiers

2-amino-ethanol (141-43-5)

N,N-didecyl-N-methyl-poly(oxethyl) ammonium propionate (94667-33-1)

N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine (2372-82-9)

1.2. Explanation of the symbols

1	CE mark with notified body
2	Medical device
3	Item number
4	Batch designation
5	Expiry date
6	Manufacturer
7	Temperature limit
8	Quantity information
9	Responsible for the translation
10	Observe instructions for use
11	Use hand protection
12	Use eye protection
13	Wear protective clothing
14	Use protective apron

15	Information
16	Protect from heat / Protect from sunlight
17	Protect from moisture / Store in a dry place
18	Fragile
19	Package orientation at the top
20	Limited quantity
21	Corrosive
22	Dangerous for the environment
23	Harmful to health
24	Unique identifier of a medical device
25	UDI marking with standard compliant HIBC data content
26	Name and address of the authorized representative's registered office in Switzerland

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended purpose

GREEN&CLEAN ID N is a concentrate intended for disinfecting pretreatment of invasive and non-invasive dental instruments

Indication: GREEN&CLEAN ID N is used for reprocessing of dental instruments after treatment.

Contraindication: There are no contraindications referring to specific patient populations or user groups. GREEN&CLEAN ID N is not applicable for the elimination of spores and prions.

Target group: Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

DANGER

H314	Causes severe skin burns and eye damage.
H373	May cause damage to organs through prolonged or repeated exposure.
H302	Harmful if swallowed.
H318	Causes serious eye damage.
H410	Very toxic to aquatic life with long lasting effects.
H400	Very toxic to aquatic life.
P101	If medical advice is needed, have product container or label at hand.
P102	Keep out of reach of children.
P260	Do not breathe dust/fume/gas/mist/vapours/spray.
P264	Wash thoroughly after handling.
P273	Avoid release to the environment.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P261	Avoid breathing dust/fume/gas/mist/vapours/spray.
P301 + P330 + P331	IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
P303 + P361 + P353	IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water (or shower).
P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor/...
P391	Collect spillage.
P405	Store locked up.
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.2.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

General instructions	Immediately remove contaminated, saturated clothing.
After inhalation	Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label. In case of inhalation of spray, consult a doctor and show the packaging or label. In case of breathing difficulties or respiratory arrest, initiate artificial respiration.
After skin contact	In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.
After eye contact	In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist.
After ingestion	Rinse mouth immediately and drink plenty of water. Do not induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.
Self-protection of the first aider	First aider: Pay attention to self-protection!
Indication of any immediate medical attention and special treatment needed	Information for the doctor Oxygen respiration if necessary. Elementary relief, decontamination, symptomatic treatment.

3.5. Personal protective equipment (PPE)

Eye/Face protection	Suitable eye protection: chemical splash goggles
Hand protection	Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm Breakthrough time: >480 min Additional hand protection measures: Wear cotton undersleeves if possible. Note: Approved protective gloves must be worn.
Physical protection	Suitable physical protection: Laboratory coat Apron
Respiratory protection	Usually no personal respiratory protection is necessary.

3.6. Combination with other products / materials

Material compatibility

Examples of typical materials for medical devices tested for material compatibility:

anodised aluminium, powder coated aluminium, nickel-plated mild steel, polished martensitic steel, gold plated stainless steel, polyethylene, polymethacrylic methacrylate, tungsten carbide-nickel composite, polyvinyl chloride coating, isobutene-isoprene rubber (two types), polycarbonate optical lenses, silicate optical lenses

Test concentration of the product: 3.0 %

Test conditions: Immersion of the material samples at 20 °C for up to 19 days.

4. Product description

4.1. Product description

Highly effective concentrate for the disinfecting, manual pretreatment of invasive and non-invasive instruments. Suitable for use in the ultrasonic unit and for thermolabile and thermostable instruments. The instruments are protected by the corrosion inhibitor contained. Free of aldehyde and phenol.

4.2. Composition

Preparation of quaternary ammonium compounds, alkylamine, non-ionic surfactants, corrosion inhibitor and water

4.3. Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
bactericidal	EN 13727	high load	5 min	2.0 %
levurocidal	EN 13624	high load	5 min	1.0 %
tuberculocidal	EN 14348	high load	60 min	1.0 %
germ carrier test bactericidal	EN 14561	high load	15 min	1.0 %
germ carrier test levurocidal	EN 14562	high load	15 min	1.0 %
germ carrier test tuberculocidal	EN 14563	high load	60 min	0.5 %
limited virucidal	EN 14476	low load	15 min	0.5 %

5. Use

5.1. Restriction of use

GREEN&CLEAN ID N is NOT suitable for the final disinfection or sterilisation of surgical instruments or endoscopes. Follow further reprocessing steps according to the instrument manufacturer's instructions, including rinsing, washing and final cleaning/sterilisation.

5.2. Dosage

Total volume	Final concentration		
	0.5 %	1.0 %	2.0 %
1 l	5 ml	10 ml	20 ml
2 l	10 ml	20 ml	40 ml
3 l	15 ml	30 ml	60 ml
4 l	20 ml	40 ml	80 ml
5 l	25 ml	50 ml	100 ml

5.3. Application

Note

- > Prepare solution with cold water only
- > Observe dosage and exposure time exactly
- > Follow the instructions of the instrument manufacturer
- > Rinse the instrument with cold water afterwards

Preparing the disinfection bath

27 Dose the GREEN&CLEAN ID N concentrate in the desired quantity.

Pour the concentrate into the disinfection bath and mix with cold water.

Cover the bath to protect the solution from contamination.

28

Observe the information in the dosage table.

Important: The solution must be replaced immediately when heavily contaminated.
The solution has a shelf life of 3 years after opening.

6. Disposal

Dispose of product in compliance with official regulations.

	before intended use	after intended use
	Product	Packaging
Product waste code	70699	150102
Hazardous waste	No	No
Waste designation	Waste not otherwise specified	Plastic packaging

7. Annex

7.1. Order numbers and scope of delivery

Order number	Designation
122000015	GREEN&CLEAN ID N 500 ml, 4 x 500 ml, measuring beaker
122000016	GREEN&CLEAN ID N 5 L, 1 x 5 L canister

7.2. Order number - Application aids

Order number	Designation
121000001	AH GREEN&CLEAN, tap canister 5 l
121000007	AH GREEN&CLEAN, ID N, measuring cup
121000002	AH GREEN&CLEAN, bur disinfection box
121000003	AH GREEN&CLEAN, disinfection tray 1.25 l

7.3. Change history

Revision	Date	Description
200005294v02	12.05.2025	"REF numbers" added on page 2.
200005294v01	10.01.2024	Corrected various spelling and translation errors.
	25.07.2023	New order numbers. Application aids added. Updated information on standing time. Information on endoscopes and surgical instruments removed (3.6, 4.1). Pictogram for „Observe instructions for use" replaced.
ZK-58.105/01	13.04.2022	Correction: Chapter 1.2 „REF number" changed to „item number"
ZK-58.105/00	22.06.2021	Newly created.

Corrected various spelling and translation errors. Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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