Planosil™

Instructions for use

EN, DE, ES, FI, FR, IT, NL, SV

Manufactured for / Hergestellt für / Fabriqué pour / Prodotto da /
Manufacturado par / Gefabriceerd voo / Tillverkad för / Valmistuttaja

PLANMECA

Planmeca Oy | Asentajankatu 6 | 00880 Helsinki | Finland
tel. +358 20 7795 500 | sales@planmeca.com | www.planmeca.com

REF 112000044

CE 0044 **MD**

DENTAL USE

EN – Instructions for use	4
DE – Gebrauchsanweisung.....	7
ES – Instrucciones de uso.....	10
FI – Käyttöohje	14
FR – Mode d'emploi	17
IT – Istruzioni per l'uso	21
NL – Gebruiksaanwijzing	25
SV – Bruksanvisning	28

1	CE 0044
2	MD
3	REF
4	LOT
5	
6	
7	
8	e

9	
10	
11	
12	
13	
14	
15	
16	

17	
18	UDI
19	HIBC
20	CH REP

EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.


1. Notes

1.1. General instructions

DENTAL USE

Storage temperature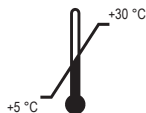

Product identifiers

Hydrogen peroxide [7722-84-1]

Silver nitrate [7761-88-8]

1.2. Explanation of the symbols

1	CE mark with notified body	13	Protect from heat / Protect from sunlight
2	Medical device	14	Protect from moisture / Store in a dry place
3	Item number	15	Fragile
4	Batch designation	16	Package orientation at the top
5	Expiry date	17	Stacking limit
6	Manufacturer	18	Unique identifier of a medical device
7	Temperature limit	19	UDI marking with standard compliant HIBC data content
8	Quantity information	20	Name and address of the authorized representative's registered office in Switzerland
9	Responsible for the translation		
10	Follow instructions for use		
11	Use hand protection		
12	Use eye protection		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of Planmeca Oy.

2. Intended purpose

Solution for disinfection and elimination of germs in water-bearing pipes in treatment units.

Indication: Limitation of the total germ count and elimination of *Legionella pneumophila*.

Contraindication: Not applicable.

Target group: Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP).

P101	If medical advice is needed, have product container or label at hand.
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. First aid measures

General instructions	Change contaminated, saturated clothing. In case of accident or if you feel unwell, seek medical advice immediately (show safety data sheet if possible).
After inhalation	Provide fresh air.
After skin contact	Subsequently wash off with water.
After eye contact	Rinse immediately carefully and thoroughly with eye-bath or water.
After ingestion	Rinse mouth immediately and drink 1 glass of water.

3.4. Personal protective equipment (PPE)

Suitable eye protection	Framed glasses.
Hand protection	Suitable gloves type Single-use gloves. Suitable material: NR (Natural rubber (caoutchouc), Natural latex). Recommended glove brands: EN ISO 374
Respiratory protection	Respiratory protection is not required when used properly and under normal conditions.

3.5. Combination with other products / materials

The product Planosil is used in the Compact™ treatment units from Planmeca Oy.

4. Product description

Planosil is a disinfectant for killing microbes in the water-bearing lines of the treatment unit.

4.1. Composition

The active ingredients are hydrogen peroxide and silver ions.

4.2. Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericidal against <i>Legionella pneumophila</i>	EN 13623	Yeast extract load	15 min	undiluted

5. Use

5.1. Dosage

The product is applied undiluted.

5.2. Application

The product Planosil is used in the Compact™ treatment units from Planmeca Oy. Please observe the instructions for use of the treatment unit.

6. Disposal

Dispose of product in compliance with official regulations.

	before intended use Product	after intended use Packaging
Product waste code	070600	150102
Hazardous waste	No	No
Waste designation	Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products	Plastic packaging

7. Annex

7.1. Order numbers and scope of delivery

Order number	Designation
30036267	Planosil, 6 x 1000 ml bottles

7.2. Change history

Revision	Date	Description
200005324	17.04.2024	Newly created.

Subject to printing and setting errors!

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Produkten Planosil används i behandlingsenheterna Compact™ från Planmeca Oy.
METASYS Medizintechnik GmbH