PlanClear™

Instructions for use

EN, DE, ES, FI, FR, IT, NL, SV

Manufactured for / Hergestellt für / Fabriqué pour / Prodotto da /
Manufacturado par / Gefabriceerd voo / Tillverkad för / Valmistuttaja

PLANMECA

Planmeca Oy | Asentajankatu 6 | 00880 Helsinki | Finland
tel. +358 20 7795 500 | sales@planmeca.com | www.planmeca.com

REF 112000045

CE 0044 **MD**

DENTAL USE

EN – Instructions for use	4
DE – Gebrauchsanweisung.....	8
ES – Instrucciones de uso.....	12
FR – Mode d'emploi	16
FI – Käyttöohje	20
IT – Istruzioni per l'uso	24
NL – Gebruiksaanwijzing	28
SV – Bruksanvisning	32

1		8		15	
2		9		16	
3		10		17	
4		11		18	
5		12		19	
6		13		20	
7		14		21	

EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.


1. Notes

1.1. General instructions

DENTAL USE

Storage temperature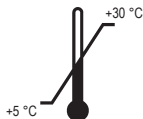

Product identifiers

Hydrogen peroxide [7722-84-1]

Silver nitrate [7761-88-8]

1.2. Explanation of the symbols

1	CE mark with notified body	13	Use eye protection
2	Medical device	14	Protect from heat / Protect from sunlight
3	Item number	15	Protect from moisture / Store in a dry place
4	Batch designation	16	Fragile
5	Expiry date	17	Package orientation at the top
6	Manufacturer	18	Stacking limit
7	Temperature limit	19	Unique identifier of a medical device
8	Quantity information	20	UDI marking with standard compliant HIBC data content
9	Responsible for the translation	21	Name and address of the authorized representative's registered office in Switzerland
10	Health hazard		
11	Follow instructions for use		
12	Use hand protection		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of Planmeca Oy.

2. Intended purpose

Solution for disinfection and elimination of germs in water-bearing pipes in treatment units.

Indication: Restriction of bioburden and elimination of *Legionella pneumophila*.

Contraindication: Not applicable.

Target group: Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

WARNING

H319	Causes serious eye irritation.
H412	Harmful to aquatic life with long lasting effects.
P101	If medical advice is needed, have product container or label at hand.
P102	Keep out of reach of children.
P273	Avoid release to the environment.
P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. First aid measures

General instructions	Change contaminated, saturated clothing.
After inhalation	Remove affected person to fresh air and keep warm and calm. If breathing is difficult or stopped, administer artificial respiration. Early administration of cortisone spray.
After skin contact	Subsequently wash off with Water.
After eye contact	In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes holding eyelids apart and consult an ophthalmologist. Protect uninjured eye.
After ingestion	Rinse mouth immediately and drink 1 glass of of water.
Self protection	First aiders: Pay attention to self-protection!

Instructions for immediate medical help	If necessary, oxygen ventilation
--	----------------------------------

3.4. Personal protective equipment (PPE)

Eye/Face protection	Suitable eye protection: Safety glasses with side protection
Hand protection	Suitable gloves type: Single-use gloves. Suitable material: NR (Natural rubber (caoutchouc), Natural latex). Recommended glove brands: EN ISO 374 Glove Material Thickness: 0,5 mm
Physical protection	Suitable physical protection: Apron
Respiratory protection	Respiratory protection is not required when used properly and under normal conditions.

3.5. Combination with other products / materials

The product PlanClear is used in the dental treatment units from Planmeca Oy.

4. Product description

PlanClear is a disinfecting concentrate for water disinfection and decontamination of treatment water and protection from limescale deposits in water pipes in treatment units.

4.1. Composition

The active ingredients for disinfecting and decontaminating the water are hydrogen peroxide and silver ions as well as phosphonobutane tricarboxylic acid to prevent limescale deposits.

4.2. Spectrum of action and exposure time

When used without patient contact

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericidal against <i>Legionella pneumophila</i>	EN 13623	Yeast extract load	60 min	undiluted
			8 hours	0.25% dilution (1 part PlanClear to 30 parts water)

For dilute application with patient contact

Spectrum of action	Standard	Test information	Exposure time	User concentration
Bactericidal against <i>Legionella pneumophila</i>	EN 13623	Yeast extract load	8 hours	150-250ppm dilution

5. Use

5.1. Dosage

PlanClear disinfectant is automatically dispensed by Planmeca's Sovereign® Classic unit.

5.2. Application

The different modes can be selected from the Planmeca Sovereign® Classic interface. Please observe the instructions for use of the treatment unit.

6. Disposal

Dispose of product in compliance with official regulations.

	before intended use Product	after intended use Packaging
Product waste code	070600	150102
Hazardous waste	No	No
Waste designation	Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products	Plastic packaging

7. Annex

7.1. Order numbers and scope of delivery

Order number	Designation
10034680	PlanClear, 6 x 1000 ml bottles

7.2. Change history

Revision	Date	Description
200005325	30.04.2024	Newly created.

Subject to printing and setting errors! .

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Produkten PlanClear används i dentala behandlingsenheter från Planmeca Oy.
METASYS Medizintechnik GmbH