EN – Instructions for use

Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C

1.2. Product identifiers

Hydrogen peroxide [7722-84-1]

Silver nitrate [7761-88-8]

1.3. Explanation of the symbols

- 1 - CE mark with notified body
- 2 - Medical device
- 3 - Item number
- 4 - Batch designation
- 5 - Expiry date
- 6 - Manufacturer
- 7 - Follow instructions for use
- 8 - Temperature limit
- 9 - Quantity information
- 10 - Responsible for the translation
- 11 - Use hand protection
- 12 - Use eye protection
- 13 - Protect from heat / Protect from sunlight

14 - Protect from moisture / Store in a dry place

15 - Fragile

16 - Package orientation at the top

17 - Stacking limit

18 - Name and address of the authorized representative's registered office in Switzerland

19 - Unique identifier of a medical device

20 - UDI marking with standard compliant HIBC data content.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of ULTRADENT Dental-Medizinische Geräte GmbH & Co. KG.

2. Intended use

Solution for disinfection and killing germs in water-bearing lines of treatment units.

Indication: Limitation of total bacterial count and elimination of Legionella pneumophila.

Contraindication: Not applicable.

User target group: Professionally trained health workers.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP).

If medical advice is needed, have product container or label at hand.

Dispose of contents/container to a suitable recycling or disposal facility.

Avoid contact with skin and eyes. Keep away from children. Do not ingest.

3.3. First aid measures

General instructions:

Change contaminated, saturated clothing. In case of accident or if you feel unwell, seek medical advice immediately (show safety data sheet if possible).

After inhalation:

Provide fresh air.

After skin contact:

Subsequently wash off with Water.

After eye contact:

Rinse immediately carefully and thoroughly with

eye-bath or water.

After ingestion:

Rinse mouth immediately and drink 1 glass of water.

3.4. Personal protective equipment (PPE)

Eye/Face protection:

Framed glasses.

Hand protection:

Suitable gloves type Single-use gloves. Suitable material: NR (Natural rubber (caoutchouc), Natural latex). Recommended glove brands: EN ISO 374.

3.5. Combination with other products / materials

The ULTRADES V product is used in combination with dental units from ULTRADENT that have a water disinfection system.

4. Product description

Disinfectant for treatment water ULTRADES V is a hydrogen peroxide-based solution for water disinfection and decontamination and for the pro-

Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericide against Legionella pneumophila	EN 13623	Low load	15 hours	1%

tection of limescale deposits in the water-carrying pipes of treatment units.

4.1. Composition

2 % hydrogen peroxide, less than 0.1 % stabilisers and silver

5. Use

Dosage:

The disinfectant ULTRADES V is automatically dosed by the disinfection system.

Application:

Suitable for ULTRADENT treatment units. Please observe the instructions for use of the disinfection system or treatment unit.

6. Disposal

Dispose of product in compliance with official regulations.

- before intended use:

Waste code: 070600; Hazardous waste: No; Waste designation: Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and

personal care products

- after intended use:

Waste code Packaging 070600; Hazardous waste: No; Waste designation: Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products

7. Annex

7.1. Order numbers and scope of delivery

8906 - 6 x ULTRADES V, 1000 ml

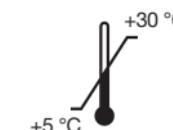
7.2. Change history

Revision 200005611, 21.08.2023: Newly created.

Corrected various spelling and translation errors.

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METASYS Medizintechnik GmbH