
DENTAL USE

CE 0044

MD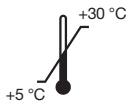

REF

112000043

XO WATER DISINFECTION

Instructions for use

DE, EN, FR, IT, DA, EL, ES, FI, NL, NO, PT, SV

200005609 | 2023-08

Table of contents

DE – Gebrauchsanweisung.....	4
EN – Instructions for use	6
FR – Mode d'emploi	8
IT – Istruzioni per l'uso	10
DA – Brugsanvisning	12
EL – Οδηγίες χρήσης	14
ES – Instrucciones de uso.....	16
FI – Käyttöohje	18
NL – Gebruiksaanwijzing	20
NO – Bruksanvisning	22
PT – Manual de utilização	24
SV – Bruksanvisning	26

Symbols

1 	7 	13 	19 
2 	8 	14 	20 
3 	9 	15 	
4 	10 	16 	
5 	11 	17 	
6 	12 	18 	

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Abbildungen:

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EN – Instructions for use

Translation of the original operation instructions

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C

1.2. Product identifiers

Hydrogen peroxide [7722-84-1]
Silver nitrate [7761-88-8]

1.3. Explanation of the symbols

- 1 - CE mark with notified body
- 2 - Medical device
- 3 - Item number
- 4 - Batch designation
- 5 - Expiry date
- 6 - Manufacturer
- 7 - Follow instructions for use
- 8 - Temperature limit
- 9 - Quantity information
- 10 - Responsible for the translation
- 11 - Use hand protection
- 12 - Use eye protection
- 13 - Protect from heat / Protect from sunlight

- 14 - Protect from moisture / Store in a dry place
- 15 - Fragile
- 16 - Package orientation at the top
- 17 - Stacking limit
- 18 - Name and address of the authorized representative's registered office in Switzerland
- 19 - Unique identifier of a medical device
- 20 - UDI marking with standard compliant HIBC data content.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of XO CARE A/S.

2. Intended use

Solution for disinfection and killing germs in water-bearing lines of treatment units.

Indication: Limitation of total bacterial count and elimination of *Legionella pneumophila*.

Contraindication: Not applicable.

Target group: Patients and trained medical personnel, without restriction as to gender and age, who come into contact with treatment water from treatment units.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP).

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

General instructions:

Change contaminated, soaked clothing. In case of accident or if you feel unwell, seek medical advice immediately (show operating instructions or safety data sheet if possible).

After inhalation:

Ensure fresh air.

After skin contact:

Then rinse with water.

After eye contact:

Immediately rinse carefully and thoroughly with eye wash or with water.

After ingestion:

Rinse mouth immediately and drink 1 glass of water.

3.4. Personal protective equipment (PPE)

Eye/Face protection:

Safety glasses

Hand protection:

Disposable gloves. Suitable material: NR (natural rubber, natural latex). Recommended glove brands: EN ISO 374.

Respiratory protection:

Respiratory protection is not required when used properly and under normal conditions.

3.5. Combination with other products / materials

Only in combination with XO Care dental units.

4. Product description

XO WATER DISINFECTION is a hydrogen peroxide-based solution for water disinfection and decontamination and for the protection of limescale deposits in the water-carrying pipes of treatment units. Contamination by Legionella pneumophila is prevented by controlled dosing of the product in the service water.

Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericide against Legionella pneumophila	EN 16623	yeast extract load	15 hours	1%

4.1. Composition

2.35% hydrogen peroxide, less than 0.1% stabilisers and silver, 98% water.

5. Use

Dosage:

The disinfectant XO WATER DISINFECTION is dosed automatically by the dental unit.

Application:

Suitable for XO Care treatment units. Please observe the instructions for use of the treatment unit.

6. Recycling and disposal

Dispose of product in compliance with official regulations.

- before intended use:

Product waste code: 070600; Hazardous waste: No; Waste designation: Wastes from the MFSU of fats, grease, soaps, detergents, disinfectants and cosmetics

- after intended use:

Product waste code Packaging 070600; Hazardous waste: No; Waste designation: Wastes from the MFSU of fats, grease, soaps, detergents, disinfectants and cosmetics

7. Annex

7.1. Order numbers and scope of delivery

AO-980 - 6 x XO WATER DISINFECTION, 600 ml

7.2. Change history

Revision 200005609, 21.08.2023: Newly created.

Corrected various spelling and translation errors. Subject to printing and setting errors!

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Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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METASYS Medizintechnik GmbH