GREEN&CLEAN IK

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

UN 1950

Storage temperature: +5 °C - +30 °C

Product identifiers: 1-propanol [71-23-8], 2-propanol [67-63-0], 2-butanol [78-92-2]

1.2. Explanation of the symbols

1	CE mark with notified body	15	Protect from heat / Protect from sunlight
2	Medical device	16	Protect from moisture / Store in a dry place
3	Item number	17	Fragile
4	Batch designation	18	Package orientation at the top
5	Expiry date	19	Limited quantity
6	Manufacturer	20	Health hazard
7	Temperature limit	21	Highly inflammable / highly flammable
8	Quantity information	22	Corrosive
9	Responsible for the translation	23	Name and address of the authorized representative's registered office in Switzerland
10	Observe instructions for use	24	Unique identifier of a medical device
11	Use hand protection	25	UDI marking with standard compliant HIBC data content
12	Use eye protection		
13	General warning sign		
14	Information		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended purpose

GREEN&CLEAN IK is a ready-for-use alcoholic disinfection spray for the spray-water and spray-air channels of dental hand pieces, contra-angles and turbines.

Indication: Reprocessing through rinsing with alcoholic disinfectant with designed and fixed mounted adapters.

Contraindication: Not applicable.

Intended users: Professionally trained healthcare personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

DANGER

H224	Extremely flammable liquid and vapour.
H318	Causes serious eye damage.
H336	May cause drowsiness or dizziness.
P101	If medical advice is needed, have product container or label at hand.
P102	Keep out of reach of children.
P210	Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P233	Keep container tightly closed.
P271	Use only outdoors or in a well-ventilated area.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor/...
P403 + P235	Store in a well-ventilated place. Keep cool.
P405	Store locked up.
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient.

Danger and safety information for the user, see point 3.2.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

After inhalation	After inhalation of spray, seek medical advice immediately and show the packaging or label.
After skin contact	After contact with skin, wash immediately with plenty of soap and water.
After eye contact	If the product gets into the eyes, rinse immediately with plenty of water for at least 5 minutes with eyes open. Then consult an ophthalmologist. Protect uninjured eye.
After ingestion	Do not induce vomiting. Rinse mouth immediately and drink plenty of water. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.
Self-protection of the first aider	First aider: Pay attention to self-protection!

3.5. Personal protective equipment (PPE)

Protection and hygiene measures	Do not eat, drink, smoke or take snuff at the workplace. Avoid contact with skin, eyes and clothing. Soiled garments must be washed before reuse.
Eye/Face protection	Suitable eye protection: Safety glasses.
Hand protection	Suitable glove type: Disposable gloves.

4. Product description

GREEN&CLEAN IK is a ready-to-use disinfectant for decontaminating the spray water and spray air channels of hand and angle pieces as well as turbines.

4.1. Composition

Alcohol-based disinfecting solution.

4.2. Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time
bactericidal	EN 13727	high load	1 min
levurocidal	EN 13624	high load	1 min
tuberculocidal	EN 14348	high load	1 min
germ carrier test bactericidal	EN 14561	high load	2 min
germ carrier test levurocidal	EN 14562	high load	1 min
germ carrier test tuberculocidal	EN 14563	high load	1 min
limited virucidal	EN 14476 EN 16777	high load	30 sec

5. Use

5.1. Application

Rinse the transfer instrument with water after use on the patient. **27:** Place the hand piece, angle piece or turbine in a paper towel to avoid aerosols. **28:** Place the hand piece, angle piece or turbine on the adapter and flood the spray water and spray air channels completely with GREEN&CLEAN IK.

The spray can is under pressure!
Do not operate the spray head until the transfer instrument is properly attached.

Apply over a wash basin.

5.2. Shelf life after opening

If stored at the temperatures specified in chapter 1.1, the product has a shelf life of 2 years.

6. Care and maintenance

6.1. Regular cleaning measures

The application of GREEN&CLEAN IK does not replace the necessary care process with oil, but can only precede it.

Lubrication of the O-rings

- 29+30** To lubricate the O-rings, place the GREEN&CLEAN IK adapter in the palm of your hand (wear gloves), spray briefly and wet the palm with the liquid.

7. Disposal

Dispose of product in compliance with official regulations.

	before intended use	after intended use
	Product	Packaging
Product waste code	070699	070699
Hazardous waste	No	No
Waste designation	Waste not otherwise specified	Waste not otherwise specified

8. Annex

8.1. Order numbers and scope of delivery

Order number	Designation	Scope of delivery
122000017	GREEN&CLEAN IK, coupling straight/contra-angle handpieces	2x 200 ml
122000018	GREEN&CLEAN IK, turbine coupling NSK	2x 200 ml
122000019	GREEN&CLEAN IK, turbine coupling NSK Quick	2x 200 ml
122000020	GREEN&CLEAN IK, coupling Dentsply Sirona T1 Classic	2x 200 ml
122000021	GREEN&CLEAN IK, turbine coupling Bien Air	2x 200 ml
122000022	GREEN&CLEAN IK, turbine coupling Castellini	2x 200 ml
122000023	GREEN&CLEAN IK, turbine coupling KaVo	2x 200 ml
122000024	GREEN&CLEAN IK, turbine coupling Dentsply Sirona	2x 200 ml
122000025	GREEN&CLEAN IK, turbine coupling W&H	2x 200 ml

8.2. Change history

Revision	Date	Description
200005322v02	12.01.2026	Under 1.1, „Dental Use“ deleted; target group and shelf life information updated. Changed: Product designation (chapter 8.1)
200005322v01	16.09.2024	Updated order numbers. Additional pictograms: UDI, CH-REP, HIBC.
200005322	05.10.2023	Newly created.

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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