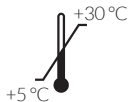
UN 3175

CE 0044

MD

GREEN&CLEAN WD

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Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C
UN 3175

1.2. Product identifiers

Ethanol [64-17-5], 2-propanol [67-63-0], Glyoxal [107-22-2], Didecyltrimethylammonium chloride [7173-51-5]

1.3. Explanation of the symbols

1 - CE mark with notified body; **2** - Medical device;
3 - Item number; **4** - Batch designation; **5** - Name and address of the authorized representative's registered office in Switzerland; **6** - Unique identifier of a medical device; **7** - UDI marking with standard compliant HIBC data content; **8** - Expiry date; **9** - Manufacturer; **10** - Follow instructions for use; **11** - Temperature limit **12** - Responsible for the translation; **13** - Health hazard; **14** - Highly inflammable / highly flammable; **15** - Use hand protection; **16** - Use eye protection; **17** - Protect from heat / Protect from sunlight; **18** - Protect from moisture / Store in a dry place; **19** - Package orientation at the top; **20** - Limited quantity; **21** - Stacking limit; **22** - Information.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended purpose

Disinfectant wipes for disinfecting alcohol-insensitive surfaces of non-invasive medical devices.

Indication: Disinfection of alcohol insensitive surfaces of non-invasive medical devices.

Contraindication: Not for usage on materials made of polymethyl methacrylate (acrylic glass), leather and synthetic leather covers.

User target group: Professionally trained healthcare personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

WARNING: Flammable liquid and vapour. Causes serious eye irritation. Harmful to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children.

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.4 - First aid measures.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

General instructions:

In case of accident or if you feel unwell, seek medical advice immediately (show operating instructions or safety data sheet if possible).

After inhalation:

Ensure fresh air.

After skin contact:

Then rinse with: Water and soap.

After eye contact:

Immediately rinse carefully and thoroughly with eye wash or with water.

After ingestion:

If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

3.5. Personal protective equipment (PPE)

Eye/Face protection:

Suitable eye protection: Tightly sealing safety goggles;

Hand protection:

Suitable material: NBR (nitrile rubber). Required properties: liquid-tight. Penetration time (maximum wearing time): <8 h. Thickness of the glove material: 0.1 mm. Wearing time with occasional contact (splashes): <30 min.

4. Product description

Ready-to-use, alcohol wipe disinfection.

4.1.Composition

One wipe contains 730 mg ethanol, 245 mg isopropyl alcohol, 3 mg quaternary ammonium salts, 3 mg glyoxal, 645 mg exp.

5. Use

The surfaces to be disinfected are wiped until completely wetted and allowed to dry for the duration of the exposure time.

6. Care and maintenance

6.1.Reprocessing the dispenser incl. lid

Remove any residue from the dispenser before refilling.

1. Remove residue mechanically.
2. Rinse the dispenser incl. lid completely (inside and out) with hot tap water (danger of scalding!) and dry with a cloth.
3. Then completely wet the dispenser incl. lid with a cloth soaked in disinfectant solution (e.g. with GREEN&CLEAN SD or SK) and disinfect thoroughly.
4. Pay attention to the application time and complete

drying.

6.2.Refilling the dispenser container

Before refilling, thoroughly clean and disinfect the dispenser container and lid. Open the refill bag and place the wipes in the dispenser. Transfer the batch number and expiry date to the enclosed label, fill it out completely, and attach it to the dispenser. Observe the safety instructions and further information in the GREEN&CLEAN WD instructions for use.

7. Disposal

Dispose of product in compliance with official regulations.

- before intended use:

Product waste code: 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

- after intended use:

Product waste code Packaging 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

8. Annex

8.1.Order numbers and scope of delivery

122000055 - GREEN&CLEAN WD 120 wipes, 6

pouches

122000079 - GREEN&CLEAN WD, 2 dispensers empty, 6 pouches

121000012 - AH GREEN&CLEAN, WD, dispenser empty, 2 pcs

8.2. Change history

Revision 200005328v02, 30.01.2026: 'Dental Use' deleted; target group updated; new order numbers; instructions for refilling the dispenser can added.

Revision 200005328v01, 20.01.2025: In chapter 1.1 General information, pictogram „Observe instructions for use“ replaced.

Revision 200005328, 04.07.2023: Newly created. Corrected various spelling and translation errors. Subject to printing and setting errors!

Spectrum of action and exposure time	Standard	Test information	Spectrum of action and exposure time
bactericidal	EN 13727	high load	30 sec
levurocidal	EN 13624	high load	30 sec
tuberculocidal	EN 14348	high load	30 sec
4-field test (bactericidal, levurocidal)	EN 16615	high load	1 min
limited virucidal	EN 14476 EN 16777	high load	30 sec

Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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