GREEN&CLEAN WK

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EN – Instructions for use

Translations

Translation of the original operation instruction

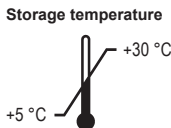
Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str.
44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Product identifiers

Hydrogen peroxide [7722-84-1], Silver nitrate [7761-88-8]

1.2. Explanation of the symbols

1 - CE mark with notified body, 2 - Medical device, 3 - Unique identifier of a medical device, 4 - Item number, 5 - UDI marking with standard compliant HIBC data content, 6 - Batch designation, 7 - Expiry date, 8 - Manufacturer, 9 - Name and address of the authorized representative's registered office in Switzerland, 10 - Follow instructions for use, 11 - Temperature limit, 12 - Quantity information, 13 - Responsible for the translation, 14 - Use hand protection, 15 - Use eye protection, 16 - Protect from heat / Protect from sunlight, 17 - Protect from moisture / Store in a dry place, 18 - Fragile, 19 - Package orientation at the top, 20 - Stacking limit, 21 - Information.

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik.

2. Intended purpose

Solution for disinfection and elimination of germs in water-bearing pipes in treatment units.

Indication: Limitation of the total germ count and elimination of *Legionella pneumophila*.

Contraindication: Not applicable.

Target group: Professionally trained healthcare personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP). If medical advice is needed, have product container or label at hand. Dispose of contents/container to a suitable recycling or disposal facility.

3.3. First aid measures

General instructions

Change contaminated, saturated clothing. In case of accident or if you feel unwell, seek medical advice immediately (show safety data sheet if possible).

After inhalation

Provide fresh air.

After skin contact

Subsequently wash off with Water.

After eye contact

Rinse immediately carefully and thoroughly with eye-bath or water.

After ingestion

Rinse mouth immediately and drink 1 glass of of water.

3.4. Personal protective equipment (PPE)

Protection and hygiene measures

Avoid contact with skin, eyes and clothing.

Eye/Face protection

Framed glasses.

Skin protection

Suitable gloves type Single-use gloves.

Suitable material: NR (Natural rubber (caoutchouc), Natural latex).

Recommended glove brands: EN ISO 374

3.5. Combination with other products / materials

The disinfectant GREEN&CLEAN WK is used together with the product WEK Water Disinfection System and WEK Light Water Disinfection System.

4. Product description

GREEN&CLEAN WK is a solution for disinfection and elimination of germs in water-bearing pipes in treatment units with hydrogen peroxide as active ingredient.

4.1. Composition

The active ingredients are hydrogen peroxide and silver nitrate.

5. Use

The disinfectant GREEN&CLEAN WK is used together with the dosing devices WEK water disinfection systems from METASYS. If the user does not have one of the devices, a bottle system can also be used.

5.1. Application with METASYS WEK Water Disinfection System

With the METASYS WEK or WEK Light Water Disinfection Systems, the disinfectant is automatically dosed into the water-bearing pipes.

Dosage:

The dosing is automated in two programs:

- > Normal corresponds to a dosing of 1 part GREEN&CLEAN WK to 85 parts water
- > Intensive corresponds to a dosing of 1 part GREEN&CLEAN WK to 42 parts water

Application:

The correct application is described in the corresponding chapter of the instruction manual (IFU) of the WEK and WEK Light water disinfection systems.

5.2. Application in the Bottle System

Dosage:

Water	Standard disinfection	Intensive disinfection
0.5 l	6 ml	12 ml
1 l	12 ml	24 ml
2 l	24 ml	48 ml
3 l	36 ml	72 ml

Note:

- > for continuous decontamination, GREEN&CLEAN WK must be added again each time the water bottle is refilled
- > Always wear gloves and protective goggles when attaching the dosing attachment and during application. Prevent contamination of the dosing aid.

Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericide against Legionella pneumophila	EN 13623	Low load	24 hours	1.18%

Preparation for application:

- 23

Avoid skin contact with the riser tube or other parts of the dosing aid that come into contact with the decontamination concentrate.
- 24 + 25

Remove the attachment from the bag and remove the seal of warranty.
- 26

Immediately after removal, attach the attachment together with the riser tube firmly to the GREEN&CLEAN WK bottle.

Application:

- 27

Dose the appropriate amount depending on the size of your water bottle by squeezing the bottle.
- i

It is a soft bottle. Therefore, only press it together lightly. Only dispense when the sealing cover is open.
- i

The dosing aid is only suitable for 750 ml GREEN&CLEAN WK bottles - a measuring cup must be used for the 1 litre GREEN&CLEAN WK bottles.
- 28

Empty the dosed amount of solution into your water bottle and fill up with drinking water to the desired volume. You can then connect the bottle to the unit.
- 29

Close the cover of the attachment after each use to protect the contents from contamination.

5.3. Shelf life after opening

If stored at the temperatures specified in chapter 1.1, the product has a shelf life of 3 years.

6. Care and maintenance

6.1. Regular cleaning measures

Dosing aid:

When changing the bottle, unscrew the attachment from the empty bottle and reattach it directly to the full bottle. Change the attachment at regular intervals (about every six months or after changing the bottle about three times) and especially when there is visible contamination.

- > when the content of the GREEN&CLEAN WK bottle is used up, remove the dosing aid from the empty bottle and attach it directly to the new bottle
- > the dosing aid must be replaced every 6 months and in case of heavy soiling
- > The dosing aid is not suitable for reprocessing.

7. Disposal

Dispose of product in compliance with official regulations.

Product - before intended use:

Product waste code: 070600

Hazardous waste: No

Waste designation: Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products

Packaging - after intended use:

Waste code Packaging: 150102

Hazardous waste: No

Waste designation: Plastic packaging

8. Annex

8.1. Order numbers and scope of delivery

Order number	Designation
122000056	GREEN&CLEAN WK 1000 ml, 6 x 1000 ml
122000057	GREEN&CLEAN WK 750 ml, 4 x 750 ml
122000077	GREEN&CLEAN WK 750 ml, 2 x 750 ml
91.00.66.00	A-dec Maintain Solution (case of 4)

8.2. Application aids

Order number	Designation
121000013	AH GREEN&CLEAN, WK, dosing cup
121000014	AH GREEN&CLEAN, WK, dosing aid for Bottle Systems, for 750 ml bottle
121000015	AH GREEN&CLEAN, WK, test sticks, 25 pcs.
121000016	AH GREEN&CLEAN, WK, test sticks, 5 pcs.
91.00.68.00	A-dec Maintain Test Strips, pkg of 5 test strips
91.00.67.00	A-dec Maintain Test Strips, pkg of 25 test strips

8.3. Change history

Revision	Date	Description
200004290 (ZK-58.106/00)	17.10.2022	Newly created.
200004290v01	17.07.2023	Updated: Intended use, order numbers.
200004290v02	28.10.2024	Update: Product description. Information added: Chapter 6.3 Shelf life after opening.
200004290v03	21.10.2025	Added: A-dec products; under 1.1, „Dental Use“ deleted; target group updated

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str.
44, 81379 Munich, Germany

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METASYS Medizintechnik GmbH