METASAN® P

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EN – Instructions for use

Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images: The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C

1.2. Product identifiers

Hydrogen peroxide [7722-84-1]

Silver nitrate [7761-88-8]

1.3. Explanation of the symbols

1	CE mark with notified body
2	Medical device
3	Unique identifier of a medical device
4	UDI marking with standard compliant HIBC data content
5	Item number
6	Batch designation
7	Expiry date
8	Manufacturer
9	Follow instructions for use
10	Temperature limit
11	Quantity information
12	Responsible for the translation
13	Use hand protection
14	Use eye protection
15	Protect from heat / Protect from sunlight
16	Protect from moisture / Store in a dry place
17	Fragile
18	Package orientation at the top
19	Stacking limit
20	Name and address of the authorized representative's registered office in Switzerland
21	Information

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik.

2. Intended Purpose

Solution for disinfection and elimination of germs in water-bearing pipes in treatment units. **Indication:** Limitation of total bacterial count and elimination of *Legionella pneumophila*. **Contraindication:** Not applicable. **Target group:** Professionally trained healthcare personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP). If medical advice is needed, have product container or label at hand. Dispose of contents/ container to a suitable recycling or disposal facility.

3.3. First aid measures

General instructions: Change contaminated, saturated clothing. In case of accident or if you feel unwell, seek medical advice immediately (show safety data sheet if possible).

After inhalation: Provide fresh air.

After skin contact: Subsequently wash off with water.

After eye contact: Rinse immediately carefully and thoroughly with eye-bath or water.

After ingestion: Rinse mouth immediately and drink 1 glass of water.

3.4. Personal protective equipment (PPE)

Protection and hygiene measures:

Avoid contact with skin, eyes and clothing.

Eye/Face protection:

Framed glasses.

Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time	Solution ratio
Bactericide against <i>Legionella pneumophila</i>	EN 13623	low yeast extract load	1 hour	10%
			24 hours	1%

Hand protection:

Suitable gloves type: Single-use gloves.
Suitable material: NR (natural rubber (caoutchouc), natural latex).
Recommended glove brands: EN ISO 374

Respiratory protection:

Respiratory protection is not required when used properly and under normal conditions.

3.5. Combination with other products / materials

METASAN® P is used in combination with treatment units from Dentsply Sirona ®.

4. Product description

METASAN® P is a disinfectant concentrate for disinfecting water and decontaminating process water.

4.1. Composition

<1.5% hydrogen peroxide, less than 0.1% stabilizers and silver

5. Use

5.1. Dosage

The disinfectant METASAN® P is dosed automatically by the Dentsply Sirona treatment unit.
If a bottle solution is used, the following dosages should be used:

Final volume	Volume of METASAN® P	Volume of water
1 liter (1% solution ratio)	10 ml	990 ml
1 liter (10% solution ratio)	100 ml	900 ml

5.2. Application

Suitable for Dentsply Sirona treatment units. Please observe the instructions for use of the treatment unit.

5.3. Shelf life after opening

If stored at the temperatures specified in chapter 1.1, the product has a shelf life of 3 years.

6. Disposal

Dispose of product in compliance with official regulations.

before intended use	
Waste code - Product	070600
Hazardous waste	No
Waste designation	Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products
after intended use	
Waste code - Packaging	150102
Hazardous waste	No
Waste designation	Plastic packaging

7. Annex

7.1. Order numbers and scope of delivery

122000059 - METASAN® P, 6 x 1000 ml

7.2. Change history

Revision ZK-58.105/00, 08.06.2022: Newly created.
Revision 200005330v01, 27.07.2023: Updated: Intended use, order numbers.
Revision 200005330v02, 24.05.2024: Symbol „Follow instructions for use“, Target group: „Professionally trained healthcare professionals.
Revision 200005330v03, 30.01.2025: Information added: Chapter 5.3 Shelf life after opening.
Revision 200005330v04, 12.11.2025: „Dental use“ deleted; target group in English corrected.
Corrected various spelling and translation errors. Subject to printing and setting errors!

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Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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METASYS Medizintechnik GmbH