GREEN&CLEAN BR

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EN – Instructions for use

Translation of the original operation instruction:

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images: The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

Storage temperature: +5 - 30 °C

1.2. Product identifiers

Hydrogen peroxide [7722-84-1]

1.3. Explanation of the symbols

1 - CE mark with notified body; 2 - Medical device; 3 - Unique identifier of a medical device; 4 - UDI marking with standard compliant HIBC data content; 5 - Item number; 6 - Batch designation; 7 - Expiry date; 8 - Manufacturer; 9 - Name and address of the authorized representative's registered office in Switzerland; 10 - Observe instructions for use; 11 - Temperature limit; 12 - Quantity information; 13 - Use hand protection; 14 - Use eye protection; 15 - Protect from heat / Protect from sunlight; 16 - Protect from moisture / Store in a dry place; 17 - Fragile; 18 - Package orientation at the top; 19 - Stacking limit; 20 - Information.

1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizin-technik.

2. Intended purpose

GREEN&CLEAN BR is a hydrogen peroxide solution for biofilm removal of water-bearing pipes in treatment units.

Indication: If the water-bearing pipes of treatment units show signs of biofilm. **Contraindication:** Not for usage of hydrogen peroxide sensitive materials like brass parts or acrylonitrile butadiene rubber seals. To be used exclusively by service technicians, not by practice staff. **User target group:** Trained technical staff.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP).

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.2.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

General instructions: Immediately remove contaminated, saturated clothing.

After inhalation: Provide fresh air.

After skin contact: Wash off immediately with water..

After eye contact: Rinse immediately carefully and thoroughly with eye-bath or water.

After ingestion: Drink 1 glass of water in small sips (dilution effect).

Self-protection of the first aider:

First aider: Pay attention to self-protection!

Indication of any immediate medical attention and special treatment needed / Information for the doctor: Symptomatic treatment.

3.5. Personal protective equipment (PPE)

Eye/Face protection:

Suitable eye protection: chemical splash goggles. Recommended brands of eye protection: DIN EN 165.

Hand protection:

Suitable gloves type: Single-use gloves. Suitable material: NR (Natural rubber (caoutchouc), Natural latex). Recommended glove brands: EN ISO 374. Suitable material: NBR (nitrile rubber). Required properties: liquid-tight.

3.6. Combination with other products / materials

GREEN&CLEAN BR is used in combination with the BR System. If the treatment unit has a bottle system, this can also be used for the application in accordance with the operating instructions.

4. Product description

GREEN&CLEAN BR is a hydrogen peroxide solution for biofilm removal of water-bearing pipes in treatment units.

4.1. Composition

4% hydrogen peroxide

Spectrum of action and exposure time	Standard	Test information	Exposure time
Bactericide against Legionella pneumophila	EN 13623	yeast extract load	15 min

5. Use

5.1. Dosage

GREEN&CLEAN BR is introduced undiluted into the water-carrying lines of the treatment unit.

5.2. Application in the water bottle (bottle system)

5.2.1. Biofilm removal

Put on protective gloves and goggles before use!

Remove all water consumers (e.g. turbines, handpieces, couplings; EXCEPT air/water syringe) from the treatment unit (protection against clogging).

A - Fill the empty water bottle of the treatment center with GREEN&CLEAN BR.

IMPORTANT: Before removing the biofilm, check whether the dentist element (e.g. instruments) is a stand-alone system or whether the dentist, assistant and patient elements (e.g. spittoon bowl, saliva ejector) of the treatment chair are connected to one another.

In a stand-alone system, 250 ml of GREEN&CLEAN BR are sufficient; if the chair elements are connected to each other, at least 1000 ml GREEN&CLEAN BR are required.

B - Connect the water bottle to the treatment unit and prime the water lines.

C - Open all water consumer outlets and wait until purple liquid comes out.

For treatment units without a spittoon bowl: use a suitable container (e.g. bucket or tub) to collect the escaping liquid!

D - As soon as purple liquid emerges, close all outlets and valves and observe the exposure time (30 min).

E - After the exposure time has expired, fill the emptied and washed water bottle of the treatment unit with clean water, connect it to the treatment unit and rinse the water bottle and lines completely.

F - If necessary, repeat the rinsing process until the entire GREEN&CLEAN BR has been rinsed out.

5.2.2. Intensive decontamination of the bottle

Attention: Biofilm removal (intensive germ reduction) may only be carried out by trained technicians!

For intensive disinfection of the bottle itself, fill the water bottle up to the bottleneck with GREEN&CLEAN BR. Cap the water bottle and swirl the liquid in it to completely wet the surfaces. The exposure time is 30 minutes.

Then empty the bottle and rinse it with clean water. If necessary, repeat this step several times until the entire GREEN&CLEAN BR has been rinsed out.

The intensive sterilization should be repeated at least once a year and if necessary, even if the water pipes are continuously decontaminated.

5.2.3. Application with BR System

If the spittoon bowl and the tumbler filler are not connected to a bottle system, we recommend the use of the BR System

from METASYS.

For information on application and safety instructions, see the instructions for use of the BR System.

5.2.4. Shelf life after opening

If stored at the temperatures specified in chapter 1.1, the product has a shelf life of 3 years.

6. Disposal

Dispose of product in compliance with official regulations.

• before intended use:

Waste code Product: 070600; Hazardous waste: No; Waste designation: Waste from MFSU of fats, lubricants, soaps, detergents, disinfectants and personal care products

• after intended use:

Waste code Packaging: 150102; Hazardous waste: No; Waste designation: Plastic packaging

7. Annex

7.1. Order numbers and scope of delivery

122000002 - GREEN&CLEAN BR 1000 ml, 2 bottles

7.2. Change history

Version ZK-58.109/00, 19.08.2022: Newly created.

Version 200005321v01, 30.09.2024: Intended purpose; minimum shelf life; new order numbers; product name „BR System“; biofilm removal instead of cleaning.

Version 200005321v02, 25.02.2026: Under 7.1, product designations updated. "Dental use" deleted. Target group updated (only in English).

Corrected various spelling and translation errors.

Subject to printing and setting errors!

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Traduction de la notice originale:

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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METASYS Medizintechnik GmbH