# META Tower

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## 1. Notes

### 1.1 Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

### 1.2 General instructions

METASYS can only guarantee the safety, reliability and performance of the dental device if the following instructions are adhered to:

- > The product is only to be used in accordance with the instructions for use.
- > During maintenance and service work (inspection, service, repair, replacement), only use original replacement parts.
- > All manufacturer specifications for the treatment units to which the device is connected must be observed.
- > After commissioning, complete the proof of installation and send this to METASYS in order to define the warranty period.
- > All maintenance and service work must be entered into the device logbook.
- > On request by an authorised technician, METASYS will provide all documentation that may be of use to technically qualified personnel during maintenance and service works
- > METASYS accepts no responsibility for damage that may arise due to external influences (defective installation), using incorrect information, improper use of the dental device, or maintenance and service works being carried out improperly.
- > The user must familiarise himself/herself with how to operate the dental device and ensure that the dental device is in good condition each time before operating it.

Important: Read the accompanying documents of the device carefully before installation, commissioning and use and keep them for the entire service life of the product!

### 1.3 Explanation of the symbols

1	CE mark	24	Electrostatic sensitive device / device parts
2	Item number	25	Mark of conformity per para. 22 section 4 Lower Saxony Building Regs
3	Serial number	26	Separate collection of electrical/ electronic equipment (WEEE)
4	Medical device	27	Observe instructions for use
5	Unique identifier of a medical device	28	Use hand protection
6	UDI marking with standard compliant HIBC data content	29	Use eye protection
7	Name and address of the authorized representative's registered office in Switzerland	30	Use mouth and nose protection
8	Manufacturer	31	Earth before use
9	Date of manufacture	32	Unplug from mains
10	Follow instructions for use	33	General warning sign
11	Follow instructions for use with reference to eIFU	34	Warning of electrical voltage
12	Temperature limit	35	Warning of hot surface
13	Caution / attention	36	Warning of explosive substances
14	Beware of electric shock	37	Warning of biological hazard
15	Humidity limit	38	Protect from heat / Protect from sunlight
16	Air pressure limit	39	Protect from moisture / Store in a dry place
17	Inclination / slope	40	Fragile
18	Maximum installation altitude	41	Package orientation at the top
19	Blower / fan	42	Stacking limit
20	On   Off	43	Information
21	Protective earthing	44	Maintenance / service
22	Applied part type B	45	Stop operation
23	Applied part type BF		

## Technical data

	META Tower 2		META Tower A2		META Tower 5		META Tower A5	
Power supply	230 V AC	400 V AC	230 V AC	400 V AC	230 V AC	400 V AC	230 V AC	400 V AC
Frequency	50 Hz	50 Hz	50 Hz	50 Hz	50 Hz	50 Hz	50 Hz	50 Hz
Max. current consumption	17,3 A	9,0 A	17,3 A	9,0 A	22,2 A	9,0 A	22,2 A	9,0 A
Max. power consumption	2,7 kW	2,54 kW	2,7 kW	2,54 kW	3,8 kW	3,8 kW	3,8 kW	3,8 kW
Max. ambient temperature	40° C	40° C	40° C	40° C	40° C	40° C	40° C	40° C
Operating time	100% (60%) / Compressor 70%		100% (60%) / Compressor 70%		100% (60%) / Compressor 70%		100% (60%) / Compressor 70%	
Tank capacity	30 l	30 l	30 l	30 l	30 l	30 l	30 l	30 l
Noise level	53 dB(A)	53 dB(A)	53 dB(A)	53 dB(A)	54 dB(A)	54 dB(A)	54 dB(A)	54 dB(A)
Weight	222 kg	222 kg	223 kg	223 kg	224 kg	224 kg	225 kg	225 kg
Dimensions (H x W x D) mm	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600	2130 x 600 x 600

### 1.4 Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik.

## 2. Safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

### 2.1 Safety instructions

Assembly, modifications or repairs may only be carried out by authorised qualified personnel who guarantee compliance with the EN 60601-1 standard (international standard on Medical Electrical Equipment and Systems, in particular part 1: General requirements for basic safety).

The electrical installation must comply with the regulations of the IEC (International Electrotechnical Commission).

Use in explosive or flammable locations is prohibited!

### 2.2 Warnings

<b>DANGER</b>	Warning of a danger that will directly result in serious injury or death
<b>WARNING</b>	Warning of a danger that can result in serious injury or death
<b>CAUTION</b>	Warning of a danger that can result in minor injury
<b>ATTENTION</b>	Warning of a danger that can result in extensive damage to property

### 2.3 Intended users

The device must only be used by professionally trained healthcare personnel. Installation, service and maintenance work may only be carried out by METASYS trained technicians.

## 3. Product description

The META Tower combines a suction system (EXCOM hybrid), compressed air supply (META Air) and, depending on the model, an amalgam separator (ECO II+).

### Suction

The suction system with integrated separation installed in the META Tower separates liquids and solids suctioned during dental treatment. Separation within the dental unit is therefore no longer required. This central suction system with integrated separation is designed for operation with one or more dental treatment units.

### Compressed air supply

The compressor integrated in the META Tower generates dry and filtered compressed air for the operation of dental treatment units and similar applications.

### Amalgam separation

The ECO II+ is a sedimentation amalgam separator for separating solids from wastewater, installed in the META Tower downstream of the central suction system.

### 3.1 Type overview

<b>46 META Tower 2</b>	<ul style="list-style-type: none"> <li>&gt; for use with 2 treatment units <b>or</b></li> <li>&gt; up to 3 treatment units, provided that only the small suction hose is used on 1 of the 3 treatment units</li> <li>&gt; without amalgam separator</li> </ul>
<b>47 META Tower A2</b>	<ul style="list-style-type: none"> <li>&gt; for use with 2 treatment units <b>or</b></li> <li>&gt; up to 3 treatment units, provided that only the small suction hose is used on 1 of the 3 treatment units</li> <li>&gt; with amalgam separator</li> </ul>

<b>48 META Tower 5</b>
<ul style="list-style-type: none"> <li>&gt; for use with 3 treatment units or</li> <li>&gt; up to 5 treatment units, provided that only the small suction hose is used on 2 of the 5 treatment units</li> <li>&gt; without amalgam separator</li> </ul>

<b>49 META Tower A5</b>
<ul style="list-style-type: none"> <li>&gt; for use with 3 treatment units or</li> <li>&gt; up to 5 treatment units, provided that only the small suction hose is used on 2 of the 5 treatment units</li> <li>&gt; with amalgam separator</li> </ul>

### 3.2 Type plate

**50** On the META Tower, the type plate is located on the inside of the bottom housing, next to the pressure monitor.

1	Device description	6	Manufacturer
2	Item number	7	Separate collection of electrical/electronic equipment (WEEE)
3	Serial number	8	Follow instructions for use
4	Connection data		
5	Date of manufacture		

### 3.3 Construction

<b>51 Illustration: META Tower A5</b>	
1	Suction system with dynamic separation
2	Pre-filter
3	Control unit
4	Hose connection for supply and exhaust air
5	Compressor
6	Cooler / Pre-filter
7	Dryer
8	Tank
9	Pressure switch
10	Amalgam separator (META Tower A2 / A5 only)
11	Expansion tank (META Tower A2 / A5 only)
12	Drain pressure monitoring
13	Tower fan
14	Drain pipe
15	Frequency converter (only for META Tower 5 / A5)

### 3.4 Functional description

Detailed descriptions of the functions of the devices installed in the META Tower are provided in the respective instructions for use and in the download area at [www.metasys.com](http://www.metasys.com).

#### Compressor

The compressor unit draws in ambient air via the intake filter. The intake air is compressed in an oil-free compression chamber and then routed through the cooler and the dryer into the pressure tank. The compressor is controlled by a pressure switch, which automatically starts and stops the unit depending on the pressure level in the pressure tank.

#### Dryer

The pre-dried and filtered compressed air passes through the membrane dryer, where the remaining residual moisture is removed by the membrane fibres. The dried compressed air is then routed into the pressure tank.

#### Suction system

The suction system starts automatically when the suction hose is removed from the holder on the treatment unit or when the inlet valve is opened. It provides the suction power required for operation of the treatment unit.

#### Amalgam separator (META Tower A2 / A5 only)

The ECO II+ amalgam separator is installed in the waste water line downstream of the suction system. It is used to separate and collect suctioned heavy metals and amalgam-containing particles. The operating principle is based on sedimentation.

## 4. Preparation for use

### 4.1 Transport and storage

The META Tower is shipped separately on a pallet, secured with an outer carton and plastic straps. During transport, the META Tower must be protected against moisture, dirt and extreme temperatures.

For any onward and return transports, the original packaging must be used and stored accordingly. The device must always be transported and stored in an upright position. The device must be transported to the installation site in a fully packed state, only then remove the transport lock and lift the unit from the pallet. After unpacking the device, check for completeness and possible transport damage.

<b>Transport and storage temperature</b>	-10°C – +40°C
<b>Transport and storage humidity limitation</b>	max. 70%

#### ATTENTION:

The META Tower may only be transported in an unpressurised condition. Before transport, the compressed air must be released from the tank and hoses and, if necessary, any condensate must be drained.

### 4.2 Installation requirements

<b>Operating temperature</b>	10 – 35 °C
<b>Humidity limit</b>	max. 70%
<b>Maximum installation altitude</b>	≤ 3000 m

- > The META Tower may only be installed in dry and well-ventilated rooms (recommended: air-conditioned rooms).
- > Installation may be carried out in a technical room on the same floor as the treatment unit or one floor below. To minimise noise exposure, the META Tower should preferably be installed in an adjacent room.
- > The device may only be installed on a level and sufficiently load-bearing surface in order to avoid vibrations.
- > The front side of the device must remain freely accessible at all times to ensure operation and maintenance access.
- > To ensure sufficient air circulation, a lateral clearance of at least 10 to 15 cm from adjacent objects such as cabinets or walls must be maintained.

#### DANGER:

Use in explosive or flammable locations is prohibited!

### 4.3 Installation

Read the instructions carefully before installation and commissioning!

Meet spatial requirements (see 4.2. Installation requirements)

#### ATTENTION:

Before assembling the META Tower, ensure that all fastenings and securing devices used for transport have been completely removed!

Description	Model
<b>52</b> Completely remove all packaging materials and transport securing devices (cardboard boxes, bubble wrap and strapping).	2 / A2 5 / A5
<b>52-1</b> META Tower 2 / A2	<b>52-2</b> META Tower 5 / A5
<b>53</b> Prepare the top half of the housing and attach the four handles to the front and rear of the top half.	2 / A2 5 / A5
<b>54</b> Place the upper housing section onto the lower housing section.	2 / A2 5 / A5
<b>55</b> Screw the upper and lower housing sections together using four fixing screws (M8 hexagon socket screws).	2 / A2 5 / A5
<b>56</b> Slide the compressor with the mounting plate into the housing.	5 / A5
<b>57</b> Secure the compressor to the mounting plate with four screws.	5 / A5
Make the following hose connections:	
<b>58</b> A - Connection between compressor and cooler (no. 01) B - Connection between pressure switch and pressure vessel (no. 02) C - Connection to vent pipe (no. 03) D - Connection between condensate drain pipe and pre-filter (no. 04) E - Connection between tank and pressure gauge (no. 05)	2 / A2 5 / A5
<b>59</b> Fit the hose connectors for the exhaust air pipe, suction pipe and waste water pipe and secure them with the screws provided (A – D).	2 / A2 5 / A5
<b>60</b> Connect the earth cable (yellow/green) to the housing doors of the upper and lower sections (A – C).	2 / A2 5 / A5

Inform the dentist about product function, operation, care and warranty conditions.

Fill in the installation proof and device logbook and return them to METASYS ([installation@metasys.com](mailto:installation@metasys.com))

## 4.4 Commissioning

### ATTENTION:

Ensure that all hoses for compressed air, supply and exhaust air, and wastewater are properly connected in accordance with the specifications.

### DANGER:

The META Tower must be properly connected to the existing power supply!

**61** Before plugging in the power cord, check that the main switch on the tower is in the "0" (off) position.

Also ensure that all plugs are correctly and exclusively inserted into the designated sockets

**62**

- X1 – Suction unit
- X2 – Compressor
- X3 – Cooler fan
- X4 – Tower fan
- X6 – Pressure switch

- 63**
- A – Main switch
  - B – Pressure gauge
  - C – Pressure switch
  - D – Pressure regulator
  - E – Condensate drain valve

- (1) **Activate fuses:**  
To turn on the META Tower, all fuses must first be activated.
- (2) **Operate the main switch:**  
Press the main switch (A) to the "I" position. If power is available, the switch lights up green, and the "Drain" LED indicator flashes briefly.
- (3) **Turn on the compressor:**  
The compressor is turned on by turning the pressure switch (C) to the "I" position. The compressor now starts automatically and is controlled by the pressure switch:
  - It shuts off as soon as the cut-off pressure of 7 bar is reached.
  - It starts again as soon as the pressure drops to 5 bar.
- (4) **Setting the operating pressure:**  
When the air tank is pressurized, the desired operating pressure can be set using the pressure regulator (D) by adjusting the adjustment screw:
  - Turn the adjustment screw clockwise to increase the pressure.
  - Turn it counterclockwise to decrease the pressure.
 The current pressure can be read on the pressure gauge (B).  
Once the desired air pressure is reached, press the adjustment screw down to lock in the set value.
- (5) **Draining condensate:**  
While the pressure is building up, open the condensate drain valve on the tank to collect any condensate in a collection container. Then close the valve again

### ATTENTION:

The safety valve is set to 10 bar. When the compressor is operated for the first time, the safety valve must be checked. The safety valve must not be used to empty the pressure tank, as this may impair its function.

#### Suction system:

To start the suction unit, a suction hose must be removed from the hose holder on the dental unit or the cuspidor bowl rinsing function (for wet suction systems) must be activated.

Check all hose connections and suction line connections for leaks.

The suction unit stops when the hose is returned to the hose holder and after a run-on time of approximately 60 seconds.

#### Pipe silencer (optional, order no.: 120000531):

The pipe silencer is used to reduce noise emissions in the exhaust air of the suction system. Installing the pipe silencer can significantly reduce noise levels in the exhaust air area.

The pipe silencer is designed for exhaust air pipes with a nominal diameter of DN 100 and is integrated into the exhaust air line of the suction system.

The pipe silencer must be installed according to the intended airflow direction. Ensure stress-free installation and secure fastening of the exhaust air line. The pipe silencer must not obstruct the airflow or impermissibly affect the required airflow rate of the suction system. After installation, all connections must be checked for secure fit and leak tightness.

#### Spare parts:

Assembly, modifications and repairs may only be carried out exclusively by authorised specialist personnel (see 2.1 Safety instructions)! The METASYS technical customer service is also available for further information and assistance in carrying out repairs, retrofitting, fault analyses, etc.

Information on spare parts can be found in the respective instructions for use of the individual components (EXCOM hybrid, META Air, ECO II+).

#### Connecting to other devices:

When connecting the METASYS device to other devices or systems, hazards can arise. It must therefore be ensured that no hazards arise for the user or the patient and that the environment is not affected. The specifications of the manufacturer of the device or system to be connected must be observed.

## 5. Electronics

### DANGER:

The META Tower may only be connected to the supply voltage (230 V / 400 V) using the supplied power cable.

For the 230 V version, the power cable must be permanently connected to the mains power supply.  
For the 400 V version, the META Tower must be connected to a suitable socket outlet protected by a magnetothermal circuit breaker. This socket outlet must be easily accessible so that the device can be quickly disconnected from the power supply in an emergency.

Extension cables must not be used!

- > The mains connection may only be carried out by a qualified electrician.
- > The electrical installation must be carried out in accordance with the applicable local regulations.
- > Before connecting to the mains, compare the rated voltage on the device type plate with the mains voltage.
- > A deviation of +/- 5% is permissible.
- > Grounding is always required for safety reasons! The power supply must be protected by the following protective devices:
  - a magnetothermal circuit breaker,
  - a residual current device (RCD, tripping current <30 mA)
  - and/or in accordance with the legal regulations applicable in the respective country.

#### Main switch:

The connection to the mains power supply must be made downstream of the practice main switch.

64 Fuses		
Abbreviation	Designation	Description
F1	Fuse 1	Main fuse, characteristic C2 in accordance with IEC/EN 60898-1
K1	Contactor 1	Main contactor
F2	Fuse 2	Fuse for compressor, pressure switch and cooler (fan), characteristic C13 in accordance with IEC/EN 60898-1
F3	Fuse 3	Fuse for suction unit, characteristic C13 according to IEC/EN 60898-1
K2	Contactor 2	Contactor for suction system
K3	Relay	Time relay: run-on time for suction system
F4	Fuse 4	Control fuse: for cabinet fan and pressure monitoring (drain) characteristic C4 according to IEC/EN 60898-1
T1	Transformer 1	Transformer: control system for suction unit
F5	Fuse 5	Transformer fuse
K4	Relay	Relay for pressure monitoring (wastewater)
S3	Thermostat	Thermostat for cabinet fan (set to 25°C)
65 Wiring diagram META Tower 2, 230 V		
66 Wiring diagram META Tower 5, 230 V		
67 Wiring diagram META Tower 2, 400 V		
68 Wiring diagram META Tower 5, 400 V		

## 6. Use

Detailed functional descriptions of the devices integrated in the META Tower can be found in the respective instructions for use (IFU) as well as in the download section at [www.metasys.com](http://www.metasys.com).

69 Control panel	
Main switch:	
1 –	15A switch On (I) / Off (0)
Pressure display:	
2 –	Cut-in pressure: 5.0 bar Cut-out pressure: 7.0 bar
3 –	Operating hours counter compressor
4 –	Fault indicator compressor
5 –	Operating hours counter suction unit
6 –	Fault indicator suction unit
7 –	Fault indicator drain

## 6.1 Negative pressure – frequency control (META Tower 5 / A5 only)

### Frequency control display

#### 70 EXCOM hybrid 5 control panel

After switching on the device for the first time, the display initially shows -HF-. After approximately 1 second, the display changes to 30.00.

- 71 When idle, this value flashes continuously. When a start signal is sent from the dental unit (first start command), the display changes to 0.00 and then starts showing the current motor frequency in real time. The displayed frequency value is always between 30.00 and 70.00.

**i** The factory setting of the EXCOM hybrid 5 devices is configured for a negative pressure of -180 mbar. This corresponds to a parameter value of 65.0.

## 6.2 Error messages

Assembly, modifications and repairs may only be carried out exclusively by authorised specialist personnel (see 2.1 Safety instructions)! The METASYS technical customer service is also available for further information and assistance in carrying out repairs, retrofitting, fault analyses, etc.

Error code	Error type	Possible cause
OC/OC1	Overcurrent	Start-up time too short
		Output short circuit
		Motor blocked
		Incorrect parameter settings
O.L1	Inverter overload	Load too heavy
O.L2	Motor overloaded	Load too heavy
O.E	DC overvoltage	Mains voltage too high
		Load inertia too high
		Deceleration time too short
PF0	Output phase loss	PID parameters set incorrectly
		Motor defective
		Motor cable loose
LU	Undervoltage	Inverter defective
		Mains voltage too low
OH	Overtemperature	Ambient temperature too high
		Poor ventilation
		Fan damaged
AErr	Signal error	Carrier wave frequency
		Analogue signal line disconnected
Err1	Incorrect password	Pressure sensor defective
		Password entered incorrectly
Err2	Incorrect parameter tuning	No password entered
		Incorrect motor parameters
Err5	PID parameter error	PID parameters set incorrectly
PCE	PMSM detuning error	Incorrect motor parameter measurement
		Motor too heavy
GP	Earth fault	Earth fault in motor cable
		Motor insulation defective
		Inverter fault

## 7. Care and maintenance

To ensure the expected service life of the device of 5 years, the specified cleaning and maintenance instructions as well as the specified installation requirements and operating conditions must be observed.

All required maintenance work for the devices installed in the META Tower can be found in the respective instructions for use (IFUs).

Before starting any maintenance or repair work, the META Tower must be switched off and disconnected from the supply network. The compressed air tank must be depressurized; the pressure condition must be checked on the pressure gauge.

### 7.1 Regular cleaning measures

#### Rinsing of the spittoon bowl

**After every treatment:** To remove residues from the lines and the suction system, the spittoon bowl must be operated briefly after each treatment and each suction hose must be flushed with cold water.

#### Cleaning and disinfection of the suction system

**2 x daily (noon/evening) and after surgical interventions,** disinfection must be carried out with the prescribed disinfectant and cleaning agent GREEN&CLEAN M2. Ideally, GREEN&CLEAN M2 should be used prior to the treatment unit remaining at a standstill for extended periods (lunchtime, end of the day or holidays). For information on use and safety instructions, see the GREEN&CLEAN M2 instructions for use.

#### 72 Empty the pre-filter

#### 73 Empty and clean the filter of the hose tray or the suction line

#### 74 Empty the filter of the spittoon drain or valve

**At least once per week (depending on the working method, emptying may also be required daily):**

Remove, empty and clean the filter drawer. Residues from the filter drawer may contain amalgam and must be collected in a suitable container.

#### ATTENTION:

It is not permitted to operate the ECO II+ amalgam separator without a pre-filter in the treatment units and/or the central suction system!

The amalgam-containing residues from the pre-filter must be collected in a suitable container and can be disposed of through a waste disposal company (e.g. METASYS logistics & collection GmbH).

## 7.2 Maintenance and service

Assembly, modifications and repairs may only be carried out exclusively by authorised specialist personnel (see 2.1 Safety instructions)! For further information and assistance in carrying out repairs, retrofitting, fault analyses, etc., the METASYS technical customer service is also available!

All required maintenance work for the devices installed in the META Tower is described in the respective instructions for use (IFUs) as well as in the download section at [www.metasys.com](http://www.metasys.com).

Before starting maintenance or repair work, the META Tower must be switched off and disconnected from the supply network. The compressed air tank must be depressurized; this must be checked on the pressure gauge.

The maintenance intervals must be strictly observed. Performed maintenance work must be documented in the device documentation. METASYS reserves the right to inspect these maintenance records.

Repair work exceeding regular maintenance may only be carried out by qualified specialist personnel or by the METASYS technical customer service. Only spare parts and accessories approved by the manufacturer may be used.

Measure	Interval	Model
<b>Suction system, annual maintenance:</b>	1 x yearly	META Tower 2 META Tower A2
1x 120000527 - 1-year maintenance kit for EXCOM hybrid 1/2		
<b>Suction system, 5-year maintenance:</b>	every 5 years	
1x 120000664 - 5-year service kit, META Tower A2		
<b>Compressor, 3000 operating hours:</b>	every 3000 operating hours	
120000322 - META Air/CAM 150, 3000-hour kit, valve block		
<b>Suction system, annual maintenance:</b>	1 x yearly	META Tower 5 META Tower A5
1x 120000490 - 1-year maintenance kit for EXCOM hybrid 5		
<b>Suction system, 5-year maintenance:</b>	every 5 years	
120000663 - 5-year service kit, META Tower A5		
<b>Compressor, 3000 operating hours:</b>	every 3000 operating hours	
120000319 - META Air/CAM 250, 3000-hour kit, valve block		
<b>Amalgam separator:</b>	at least once per year or when 100% full	META Tower A2 META Tower A5
See 7.3 replacement intervals ECO II+		
<b>Compressor, filter cartridges:</b>	1 x yearly	All models
1x 120000320 - META Air/CAM 150/250 filter cartridge replacement kit		
Replace exhaust air germ filter (optional installable)	1 x yearly	
Replace pre-filter	as required	
Service of the separation unit	as required	

### 7.3 Replacement intervals

The maximum service life (replacement interval) of the ECO II+ amalgam separator results from the chosen installation option and the number of operators.

Separator ECO II+	No. of operators	Max. service life (months)
1	1	12
	2	6
	3	9
2	4	6
	5	5
	6	4

The resulting exchange interval must be observed! The ECO II+ must be replaced as soon as the maximum filling level has been reached, or 1x a year even if the maximum filling level has not yet been reached.

### 7.4 Filling level

- 75 A label on the ECO II+ collection container shows the filling level (100%, 95%) The label enables the user to monitor the filling level (see Appendix: General technical approval / DIBT).

When using insoluble air abrasion powders the service life/replacement intervals need to be defined in the applicant's responsibility considering the expected consumption of the abrasion powder.

The water level must not be exceeded during operation. If this mark is exceeded during operation, a service technician must be contacted.

When the ECO II+ is not in operation, i.e. when the suction system is not running, the liquid level must be within the range of the „MIN. OPERATING WATER LEVEL“ mark. Check the operating water level daily.

### 7.5 Replacing the collection container

The ECO II+ collection container must be exchanged according to the replacement intervals as defined in the replacement interval table.

#### WARNING:

**Risk of contamination:** To avoid infection, wear personal protective equipment (hand, eye, nose and mouth protection) and disinfect and clean the device!  
Avoid contact with the contents of the amalgam separator!

It is not possible to open the ECO II+ and reuse it! Every attempt to open the amalgam separator results in the malfunction of the device. Moreover, the required separation rate will no longer be achieved!

- 76 Remove the adapter with the tube at the waste water inlet and waste water outlet.
- 77 Tightly close the outlet of the full amalgam separator with the sealing plug (supplied with the amalgam separator) and secure with a clip.
- 78 A + B - Release the tensioning straps on both sides of the container and remove the full amalgam separator upwards.  
Position the full amalgam separator horizontally onto an even surface, make sure that the inlet opening is pointing upwards!
- 79 Pour disinfectant for suction systems into the full collection container (20 ml GREEN&CLEAN M2).
- 80 Afterwards also tightly close the inlet opening with the sealing plug and secure with a clip.
- 81 Put the amalgam separator into the styrofoam shells, then place into the cardboard box.
- 82 Place the new amalgam separator on the fixture (A) and secure it with the tensioning straps on both sides (B).
- 83 Reconnect the adapter with the tube to the "IN" and "OUT" openings.

Check inlet and outlet of the ECO II+ for tightness!

### 7.6 Disposal of the full collection container

Place the properly sealed amalgam separator into the two styrofoam shells.

- 84 Put the amalgam separator together with the polystyrene inserts into the transport box (with postal certification) and close it.  
Prepare the package for shipment and dispose of it through a waste disposal company (e.g. METASYS logistics & collection GmbH).

#### WARNING:

Risk of contamination: To avoid infection, wear personal protective equipment (hand, eye, nose and mouth protection) and disinfect and clean the device!

Certificates of recycling need to be stored according to national law.

## 8. Decommissioning

### 8.1 Disassembly

#### WARNING:

Remove from power source before disassembly!

#### WARNING:

Risk of contamination: To avoid infection, wear personal protective equipment (hand, eye, nose and mouth protection) and disinfect and clean the device!

#### Decommissioning during prolonged periods of non-use

If the META Tower is not going to be used for an extended period, it is recommended that you drain the condensate from the pressure vessel and run the compressor in the META Tower for approximately 10 minutes with the condensate drain open. You should then switch off the compressor using the pressure switch, close the condensate drain and unplug the power cord.

#### Return transport of the device

If it is necessary to return the device to the depot or to METASYS, the original METASYS packaging must be used. Before packing the METASYS device to be transported, clean and disinfect it. Possible openings where residual fluids could escape must be closed.

### 8.2 Recycling and disposal

#### WARNING:

The device may be contaminated! Advise the disposal company of this so that appropriate precautions can be taken. Parts that are contaminated with amalgam, such as sieves, filters, hoses, etc., must also be disposed of in accordance with the national regulations.

Uncontaminated plastic parts of the device can be recycled as normal plastics. The integrated electronic components (including circuit board) are to be disposed of as electronic waste. Metal parts are to be disposed of as scrap metal.

Uncontaminated plastic parts of the device can be recycled as normal plastics. The integrated electronic components (including circuit board) are to be disposed of as electronic waste. Metal parts are to be disposed of as scrap metal.

Alternatively, the device can be returned to the manufacturer for proper disposal. Before packing the METASYS device to be transported, clean and disinfect it. Possible openings where residual fluids could escape must be closed. The original METASYS packaging must be used for shipping.

The proof of installation and the device logbook must be kept for 5 years after the device has been disposed of.

## 9. Order numbers and scope of delivery

See annex on page 38.

## 10. Warranty conditions

METASYS grants a guarantee of 12-36 months for specific products (duration of guarantee depends on the product according to the information in the current price list).

The guarantee covers all material faults that more than negligibly affect the function of the device. The guarantee does not cover damages caused by incorrect or improper handling as well as normal wear.

Furthermore, the guarantee does not apply to the replacement of the amalgam collection container or to fragile parts such as glass, plastic, hoses, filters, condensate filters or membranes. Any incurred working and travel times are excluded from the warranty.

In order to determine the validity of the guarantee, the installation proof accompanying the device must be returned to METASYS immediately after proper assembly has taken place. In this case the warranty period begins with commissioning. In the event of installation without returning the installation proof to METASYS, any warranty claim is forfeited. Installation and return of installation proof must be completed within 24 months from the date of sale from METASYS.

Furthermore, any warranty claims of the customer expire if only one of the following circumstances arises, regardless of whether the circumstances arise for the customer of METASYS or a later owner or operator:

- Improper installation, operation, maintenance or transport of the device. If METASYS parts have to be returned, the original METASYS packaging must be used for shipping. Prior to packaging and shipping, the METASYS device must be cleaned and disinfected. Any openings where residual fluids could leak must be closed.
- Installation and return of installation proof have not been completed in the aforementioned 24 months.
- Failure to send the Installation Proof to METASYS.
- Installation and use of non-original METASYS parts.
- Installation of the device by personnel who are neither trained nor authorized by METASYS.
- Occurrence of damages through improper handling and operation or use of unapproved cleaning and/or disinfecting material, as well as non-compliance with the instructions for use.
- Execution of repairs by unauthorised repair shops or unauthorised personnel.
- Failure to comply with the prescribed maintenance intervals. Maintenance must be carried out 11-12 / 23-24 / 35-36 months after installation of the respective METASYS item.
- Missing entries in the Equipment Logbook regarding the installation as well as prescribed servicing by technicians trained by METASYS.
- Failure to take reasonable immediate measures to avoid further damages in the event of a malfunction.
- Shipping of devices or components to METASYS without proper accompanying paperwork, in particular without error description or invoice for the purchase of the device.
- Failure to provide visual images (photos, video clips ...) of the METASYS item complained about, as well as of its installation situation and ambient conditions.

METASYS reserves the right to demand the documentation supplied with the device to check the maintenance intervals for the assertion of warranty claims. The processing of warranty claims takes place exclusively according to the following method:

In the event of malfunctions, the device must be opened by an authorized technician, the relevant component taken out and sent to METASYS unopened and cleaned. The customer sends the device or component in question to METASYS at his own expense. METASYS checks whether there is a warranty claim. METASYS will repair the device or component if it is cost-effective. The customer is charged for the costs incurred for the repair, but not the replacement parts covered by the guarantee. The consignment of the device or component to METASYS always represents a repair order for METASYS. For cost estimates for the repair of returned devices, a processing fee\* is charged if the warranty period has expired or no warranty case exists. For products inspections only without any cost estimates a processing fee\* can be charged. When sending the device or component to METASYS, an error description with all importation information about the device must always be included. The customer of METASYS may only provide payment in advance after consultation with METASYS. Only the affected component is to be sent in (smallest possible unit). If contaminated, intact parts are sent to METASYS without technical necessity, METASYS is entitled to destroy them without separate payment. The new part corresponding to the part to be destroyed is only delivered after a separate order and invoice. In any case, METASYS has the right to handle the guarantee through credit note or the return of new parts without conducting repairs. Guarantee services do not result in an extension of the warranty period nor do they initiate a new warranty period. The warranty period for installed replacement parts ends with that for the originally delivered device. The METASYS customer is obligated to make his customer aware of the conditions regarding the warranty processing. The statutory warranty rights of the customer remain unaffected.

\* The current warranty conditions and fees can be found in the current METASYS price list.

## 11. Change history

200003996v01	16.06.2026	Newly created.
--------------	------------	----------------

Subject to printing and setting errors!

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**METASYS Medizintechnik GmbH**